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GRIT

coarse/medium grit, blue/purple
fine/ultra-fine grit, pink/white

The Instructions for use are for single use medical devices and only covers the products listed in the relevant product group schedules.

M+W Flexistrips are used by the Dental Surgeon for the finishing and polishing of composite restorations.

The strips are produced in 2 different strips comprising of 4 different grits and give best performance when applied under slight pressure.

STERILISATION

The strips do not require sterilisation.

The strips should be kept in their original packaging at room temperature and protected against dust and moisture until used.

STORAGE

The strips should be kept in hygienic stands, dishes or other suitable containers and they must be protected against dust, moisture and contamination during storage. If the strips are not used right away, it is advisable to keep them in their original packaging.

PROPER USE

- Products that are deformed should not be used and must be disposed of.
- Use the strips in the correct order, incorrect procedures can result in a reduction in the polishing quality.
- Wear a face mask to prevent biological transfer.
- Incorrect use produces poor results and increases the risk. These products must only be used by qualified staff.

PRESSURE

- Excessive pressure must be avoided at all times.
- Excessive pressure may cause heat build up which may damage the strip.

Excessive pressure may cause overheating, which injures the pulp. In extreme cases, the instrument may even fracture.

INSTRUCTIONS

- 1) Insert the central section (the area without abrasive) of the finishing strip between contact points.
- 2) Position the selected portion of the strip over the composite surface to be finished.
- 3) Firmly grasp both ends of the strip and draw the abrasive over the composite in a vigorous, back and forth motion.
- 4) Repeat procedure using another grade of the strip.
- 5) Discard strip.
- 6) Repeat steps 1-5 with Polishing strips.

DISPOSE OF WORN INSTRUMENTS

Fractured and incorrectly shaped strips can cause injury.

SAFETY PRECAUTIONS

These dental instruments were developed and manufactured for their specific dental surgical application. Incorrect use may harm tissue, cause premature wear, destroy the instruments and endanger the operator, patient or third parties.

	Manufacturer
	Date of manufacture
	Authorised Representative
	Distributor
	Importer
	Instructions for use
	Lot Number
	Product Reference
	Medical Device
	Autoclave
	Quantity
	Eye Protection
	NB Number
	Maximum rpm
	Use by date
	Keep away from sunlight

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