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M+W K-File,  
M+W Reamer,  
M+W Hedstromfile

### Intended use

The product is a dental repair instrument for the removal of dentin and root canal forming, the treatment of various pulpitis, pulp necrosis of root canal disease.

### Structure and composition

The hand root canal file consists of working part, handle and Stopper.

### Main materials

#### Working part

Stainless Steel or NiTi alloy

The working part of these instruments is made of Stainless Steel or NiTi alloy

#### Handle

Plastic PBT

#### Stopper

Silicone

### Contraindication

The product contains Nickel and chromium should not be used for individuals with known allergic sensitivity to this metal.

### Warning

- This product can only be used by professional dentists;
- Ensure that the product is disinfected every time it is used;
- Do not use this product for any purpose other than dental treatment and service.

### Precautions for use

- For your own safety, please wear personal protective equipment (gloves, glasses, mask);
- Check whether the product is within the specified period of validity before use;
- To prevent infection, please disinfect the product with autoclave before use;
- This product should be treated as medical waste.

### Usage and Operation Steps

- Open the pulp crown with a diamond needle.
- Explore root canal with 8# or 10# file, use round-trip rotation to enter root canal, use sodium hypochlorite to flush root canal.
- Use a 15# file to further smooth the root canal and determine the working length.
- Flush again and use a 15# file to clean the shreds to prevent debris from clogging the root canal.
- Use 20# file to pull up and down, prepare 2-3 mm, each time to clean debris gradually to the root tip.
- Use 25# file to repair the root canal with a fine file, so that the root canal smooth, in the preparation process of pre-resistance do not force down, use sodium hypochlorite to wash the root canal many times, prevent debris to the root tip, resulting in filling failure.
- For large root canals, select the appropriate size file needle to be prepared in turn to size in turn.

### Cleaning, disinfection and sterilization

1. After use, rinse the root canal file under running water and clean any residue remaining on the file with a soft brush
2. Then, the root canal hand files should be immersed in a suitable disinfectant
3. The recommended sterilization method is 134 °C and 18min;

### Product storage conditions

1. During transportation and storage, handle with care, do not stack, to prevent damage to packaging materials;
2. Avoid storage in high temperature, high humidity and direct sunlight, and store the products at room temperature;

### Validity

The product is valid for 5 years.

## Symbol identification

K-File	H-File	Reamers	sterile	See directions for use
LOT number	Date of manufacture	Validity	Manufacturer	European representative

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