# ATTACHMENT-KLEBER NEM

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## Ⓢ Attachment-Glue NEM

### Intended use

Two-component composite for the fixation of prosthetic attachments. The product can be applied on the model or directly in the patient's mouth.

### Directions for use

Use M+W Attachment-Glue NEM to create passive fit connections of metal objects. A high proportion of inorganic fillers guarantee a high final hardness producing the stability required for laboratory preparation.

### Indications

- Fixing telescopic or conical crowns to a framework
- Fixing pre-fabricated attachments to a framework

### Constructing the joint

Design retentive and solid bonding surfaces. When placing the secondary component onto the primary component, a uniform, thin gap between them is necessary. This gap ensures that a passive fit is obtained and it must be filled with M+W Attachment-Glue NEM. In the case of pre-fabricated attachments, create undercuts in the bonding areas. Please follow the manufacturer's instructions.

### Preparing the bonding surfaces

After ensuring that both the primary and secondary components fit passively, create undercuts on them and sandblast with an aluminium oxide (max. 125 µm). After steam cleaning, condition the metal surfaces with a metal bonding system.

### Dosage

Put the mixing tip on the syringe, then push the plunger of the syringe. The two components are

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mixed automatically. After use, keep the mixing tip on the syringe as a closure.

**Working time (once the mixing has started):** approx. 3 min

**Setting starts after:** approx. 4 min 30 sec

**Setting is completed after:** approx. 8 min

These times assume a room temperature of 22 °C (71.6 °F).

### When working inside the mouth:

**Working time (once the mixing has started):** approx. 45 sec

**Settings starts after:** approx. 1 min

**Setting is completed after approx.:** 1 min 30 sec

Higher temperatures accelerate the setting reaction and lower temperatures will slow the setting.

### Application

Use a spatula to apply M+W Attachment-Glue NEM to the matrix and patrix. Reposition the metal components on the model and fix them into place creating a passive fit. Completely coat the areas to be bonded with M+W Attachment-Glue NEM. Ensure that they are free of air bubbles and contamination.

### Application inside the patient's mouth

Apply M+W Attachment Glue NEM with a spatula to the components that you would like to join. Reposition the metal components on the model and fix them into place creating a passive fit.

### Important

Do not move the components that are being joined before Link has polymerized completely.

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### Note

- Attachment-Glue NEM is dimensionally stable up to 120°C (248°F).
- Attachment-Glue NEM is easy to grind.

### Storage

If refrigerated at 3°C-9°C (37.4°F-48.2°F) this product can be kept for 2 years. Do not use after the expiry date.

### Side-effects

With proper use of this medical device, unwanted side-effects are extremely rare. Reactions of the immune system (allergies) or local discomfort, however, cannot be ruled out completely. Should you learn about unwanted side-effects – even if it is doubtful that the side-effect has been caused by our product – please kindly contact us.

### Contra-indications / interactions

If a patient has known allergies against or hypersensitivities towards a component of this product, we recommend not to use it or to do so only under strict medical supervision. In such cases, we will supply the composition of our medical device upon request. The dentist should consider known interactions and crossreactions of the product with other materials already in the patient's mouth before using the product.

### Note

Please supply the dentist with the above information, if this medical device is used to produce a special model. Please also consider the safety data sheets.

Avoid contact of unpolymerized material with skin and mucous membranes.

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## Troubleshooting

Working time is too short	High ambient temperatures caused by e.g. summer heat or central heating will decrease the working time.
Working time is too long	If you use the product directly from the refrigerator, this will increase the working time
Material does not polymerize sufficiently	• Substances containing eugenol/ oil of wintergreen inhibit polymerization • The material was not intermixed properly. Only use original mixing tips.
Material has already polymerized	After the last use, the cap was put back onto the syringe instead of leaving the mixing tip on it (contamination of components A+B)
Construction does not fit	• The objects were moved before the polymerization was completed. • Material was already too hard: Please bond fewer parts at a time.

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GDF GmbH · Dieselstr. 5-6 · 61191 Rosbach/Germany · Tel.: +49 - 6003-814-0 · Fax: +49 -6003-814-901