help:ex plaque f

Plaque Remover | REF. 6700 0100

DEUTSCH

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ENGLISH

Danger. Contains: Methanesulphonic acid, Hydrochloric acid. May be corrosive to metals. Causes severe skin burns and eye damage. Wear protective gloves and eye/face protection. IF ON SKIN (or hair): Take off immediately all contaminated clothing. Rinse skin with water or shower. IF IN EYES: Rinse cautiously with water for several minutes. Remove contact lenses, if present and easy to do. Continue rinsing. Immediately call a POISON CENTER. Absorb spillage to prevent material damage.

Product description:

Special cleaner for use in the SYMPRO high-performance cleaning device. Ready-for-use acid-based cleaning agent for easy removal of stubborn deposits such as plaque, tartar and denture adhesive from removable dental restorations.

Indication:

A cleaning agent for the removal of plaque and tartar deposits on removable dental restorations. Only to be used by trained dental personnel.

Contraindication:

Application in the mouth or on the body is not permitted. The use of the cleaning agent on restorations belonging to patients with a known allergy to individual constituents is contraindicated. The cleaning agent is not intended for multiple use. Restorations containing feldspar ceramic must not be cleaned with the product (risk: pitting).

Special hazard warnings:

Please read the cleaning agent safety data sheet before use. Download under www.renfert.com

May be corrosive to metals. Causes severe skin burns and eye damage.

Wear protective gloves and eye/face protection.

IF ON SKIN (or hair): Take off immediately all contaminated clothing. Rinse skin with water or shower. IF IN EYES: Rinse cautiously with water for several minutes. Remove contact lenses, if present and easy to do, continue rinsing. Immediately call a POISON CENTER/doctor.

The liquids must not come into contact with the following surfaces: PVC, linoleum, natural and artificial stone, marble, stainless steel, wood. In case of contamination and spillage, clean immediately with a copious amount of water in order to avoid material damage.

In case of complaints or events in connection with the medical device, please inform Renfert immediately.

Renfert will investigate the matter and report serious incidents to the authorities in the member states concerned.

Dosage and application:

Before use, it is essential to read the comprehensive Instructions for Use of the SYMPRO high-performance denture cleaning device. Download under www.renfert.com

help:ex plaque f is intended for single-use only, in order to prevent cross contamination. Before applying, any loose contaminants must be rinsed from the restoration under running water. Before and after the cleaning procedure, observe the disinfection specification according to national hygiene regulations.

- ▶ When soiled with denture adhesive deposits, roughly clean the denture using a dry disposable cloth first.
- ▶ Clean using 200 ml help:ex plaque f in high-performance cleaning device for 10 – 30 min. (SYMPRO-beaker incl. cleaning needles) depending on the soiling level.
- ▶ Details on how to operate the device can be found in the Instructions for Use for the device.
- ▶ **Note:** Remove the objects using the tweezers.
- ▶ Carefully rinse the cleaned object with water (at least 5 seconds), examine the restoration visually for damage and disinfect according to the national hygiene guidelines before returning to the patient.
- ▶ **Note:** The cleaning liquid can become contaminated with bacteria from the inserted object. For this reason: Always use fresh cleaning liquid for every new cleaning sequence! The cleaning needles can be used more than once.
- ▶ Empty the beaker:
 - ▶ Place holding magnets at the base of the cleaning beaker and carefully pour the cleaning liquid out.
 - ▶ Rinse the beaker, the beaker lid and cleaning needles with water.
 - ▶ If the denture was soiled with denture adhesive, clean the beaker including the needles thoroughly in hot water. This is the only way to remove loose denture adhesive particles from the beaker.
- ▶ Hygienic preparation of the system components:
 - ▶ Please follow the tips in the “Guidelines for hygienic preparation” in the Instructions for Use in the SYMPRO device.

Safety instructions:

The owner and user is responsible for proper disinfection of the cleaning needles, the cleaning beaker and lid as well as for compliance with national hygiene regulations. In order to avoid unintended effects, do not mix the disinfectant and cleaning liquids together.

Tips on disposal:

Please observe the national regulations for waste disposal when disposing of the used cleaner. This also applies for the introduced contaminants.

For details see SDS Download under www.renfert.com

Storage:

Store in a dry environment and protect against direct sunlight. Observe storage temperature according to product label.

Shelf life:

For the shelf life, please see the product label. Do not use once this date has expired.

Once a bottle has been opened, use within 6 months.

Constituents:

The liquid cleaner contains hydrochloric acid, methane sulfonic acid, glycol ether and < 5% nonionic tensides.

help:ex plaque f

Plaque Remover | REF. 6700 0100

ENGLISH

Disclaimer:

This product has been developed and is intended exclusively for professional use by trained dental professionals. It must be handled in accordance with the information provided in this instruction document. The manufacturer assumes no liability for damages that arise from improper handling or processing.

Unique Device Identifier

Professional use only

Observe the Instructions for Use*

EC conformity

Medical device

Manufacturer specifications

Not intended for reuse

Batch designation

Attention: observe accompanying document*

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Renfert GmbH