## 7. Finishing / Checking the occlusion

After polymerization, remove excess material with suitable tungsten carbide finishers or fine-grit diamonds.

Remove proximal excess with diamond or tungsten carbide finishers or using finishing strips or flexible finishing discs. Check the occlusion and articulation and grind in the necessary adjustments to prevent premature contacts or undesired articulation paths on the surface of the restoration.

### Application notes

- In the case of adjustments, additional Cention Forte can be directly applied to the polymerized material. If the inhibition layer has already been removed, the filling surface must be roughened and then coated with Cention Primer before fresh Cention Forte can be applied.
- Use Cention Forte at room temperature. Cold components may be difficult to mix.

## 3 Safety information

- In the case of serious incidents related to the product, please contact Ivoclar Vivadent AG, Bendererstrasse 2, 9494 Schaan/Liechtenstein, website: [www.ivoclarvivadent.com](http://www.ivoclarvivadent.com), and your responsible competent authority.
- The current Instructions for Use are available in the download section of the Ivoclar Vivadent AG website ([www.ivoclarvivadent.com](http://www.ivoclarvivadent.com)).

### Warnings

Prevent any contact of uncured Cention Forte and Cention Primer with skin / mucous membrane and eyes. If Cention Primer should accidentally come in contact with the eyes, immediately rinse with plenty of water and consult a physician or eye specialist. If the product comes in contact with the skin, wash with soap and plenty of water. Unpolymerized Cention Forte and Cention Primer may have a slight irritating effect and may lead to a sensitization against methacrylates. Commercial medical gloves do not provide protection against the sensitizing effect of methacrylates.

### Disposal information

- Remaining stocks must be disposed of according to the corresponding national legal requirements.

### Residual risks

Users should be aware that any dental intervention in the oral cavity involves a certain risk of complications. Some of these risks are listed below:

- Failure of the adhesive bond (loss of the filling)
- Inclusion of air bubbles during the placement of the filling
- Heat development during the curing procedure
- Wear of the filling
- Chipping, fractures
- Ingestion of material
- Postoperative sensitivity

## 4 Shelf life and storage

- Storage temperature 2-28 °C (36–82 °F)
- If the indicated storage temperature is exceeded, proper functioning of the product cannot be ensured.
- Keep the foil bag of the Cention Forte capsule closed until use.
- Cention Primer single-use applicators must be stored in the plastic container provided. Keep the plastic container closed.
- Do not use the product after the indicated date of expiration.
- Expiry date: See note on the packages.

## 5 Additional information

Keep material out of the reach of children!  
Not all products are available in all countries.

The material has been developed solely for use in dentistry. Processing should be carried out strictly according to the Instructions for Use. Liability cannot be accepted for damages resulting from failure to observe the Instructions or the stipulated area of use. The user is responsible for testing the products for their suitability and use for any purpose not explicitly stated in the Instructions.

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