M+W Rüttler 100/150

RÜTTLER | SHAKER

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Büdingen, 26. Mai 2021

Contents

■ 1. General Information	P. 9
■ 2. Safety Regulations	P. 9
■ 3. Device Structure & Scope of Delivery	P. 10
■ 4. Initial Operation	P. 11
■ 5. Maintenance & Care	P. 11
■ 6. Technical Data	P. 11
■ 7. Guarantee & Warranty	P. 12
■ 8. Troubleshooting	P. 12
■ 9. Disposal in Accordance with EC Directives	P. 12
■ 10. EC Declaration of Conformity	P. 13

1. General Information

The pictures, descriptions and qualities as presented in this manual are not binding. Apart from the main qualities the manufacturer reserves the right to make changes at any time in response to technical or commercial requirements.

The machine has been designed and manufactured exclusively for use in dental technology and dentistry. Purposes other than these are not intended by the manufacturer. The manufacturer consequently disclaims any and all responsibility for any damage of whatever kind caused by improper use of the machine.

Manipulations and alterations to the most minor degree and the use of non-original spare parts free the manufacturer from any and all kind of responsibility and void all warranty claims. The machine must only be operated by suitably trained adults.

2. Safety Regulations

- When unpacking the device please make sure that the device and its accessories are complete.
- Before plugging in the appliance, make sure that the electrical specifications written on the information plate correspond to your electricity supply specifications.
- Fit a plug on to the power supply that conforms to European standards or to the standards of the country where the appliance will be used.
- The socket must be earthed.

WARNING!

The earth connection potential is required by law and should be installed by qualified technicians.

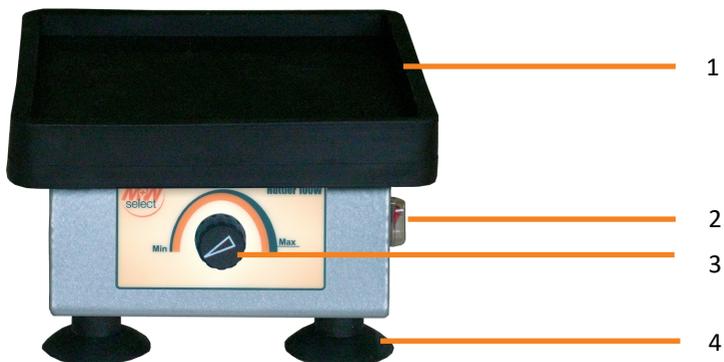
- Never use the machine if the wirings or sockets are damaged.
- Operate the machine only within spaces that are not exposed to environmental influences such as rain, snow etc.

Beware of Hazardous voltage! Emblems of warning are safety emblems which advise against risk or danger (DIN 4844-2 T2). Danger of electricity, electric shock, arc flash labels (BGV A 8 app.2 W08).

3. Device Structure & Scope of Delivery

- 1 Main Switch on/off
- 2 Power regulator, infinitely variable
- 3 Shaker table
- 4 Vacuum rubber feet

Scope of delivery: M+W Shaker 100/150, Technical Manual

4. Initial Operation

- Connect the shaker to the power plug.
- Switch on via the main switch (1).
- Set the required intensity (2) (infinitely variable).

5. Maintenance & Care

- Regular cleaning ensures reliable operation of the apparatus.
- For cleaning, use a slightly moistened cloth without alcohol, solvents, etc.
- Further maintenance procedures are not necessary.

WARNING!

 Only clean when the power cable is disconnected!

6. Technical Data

Modell	M+W Shaker 100	M+W Shaker 150
Power	100 W	150 W
Voltage	230 / 120 V ; 50/60 Hz	230 / 120 V ; 50/60 Hz
Current	1,5 A	2 A
Dimension vibration table	150 x 155 mm	150 x 230 mm
Dimension machine	170 x 170 x 110 mm	160 x 250 x 100 mm
Weight	3 kg	5 kg

7. Guarantee & Warranty

- 12 months warranty starting with the date of purchase.

Our application-technical recommendations are based on our own experience and tests and are only guidelines. It is the responsibility of the user to check the products supplied by us for their suitability for the intended processes and purposes. Our products are subject to continuous development. We therefore reserve the right to make changes in construction and composition. Of course we guarantee the perfect quality of our products.

IMPORTANT!

Müller & Weygandt GmbH has developed this manual with the intention of supporting the user in the use of the equipment and to facilitate the initial operation and use of the device. Müller & Weygandt GmbH is not responsible for any damage caused by improper use or modifications of the unit itself. Any misuse of the device will invalidate the warranty. Müller & Weygandt GmbH reserves the right to make changes or improvements to the products described in this documentation or the document itself without prior notice. Müller & Weygandt GmbH does not guarantee the completeness or correctness of the contents of this document.

8. Troubleshooting

Breakdowns	Possible Causes	Remedy
The machine does not start	Defective fuse	Replace the fuse

9. Disposal in Accordance with EC Directives

- This device must not be disposed of in the household waste!
- Disposal only by certified specialist companies or by the manufacturer.
- WEEE registration number: DE56598870

10. EC Declaration of Conformity

The manufacturer M+W Dental
Müller & Weygandt GmbH
Reichardsweide 40
63654 Büdingen

hereby declares that the following device

Product name: M+W Shaker 100/150


Art. No.: 1010735/ 1010734

meets all requirements of the following directives and standards:

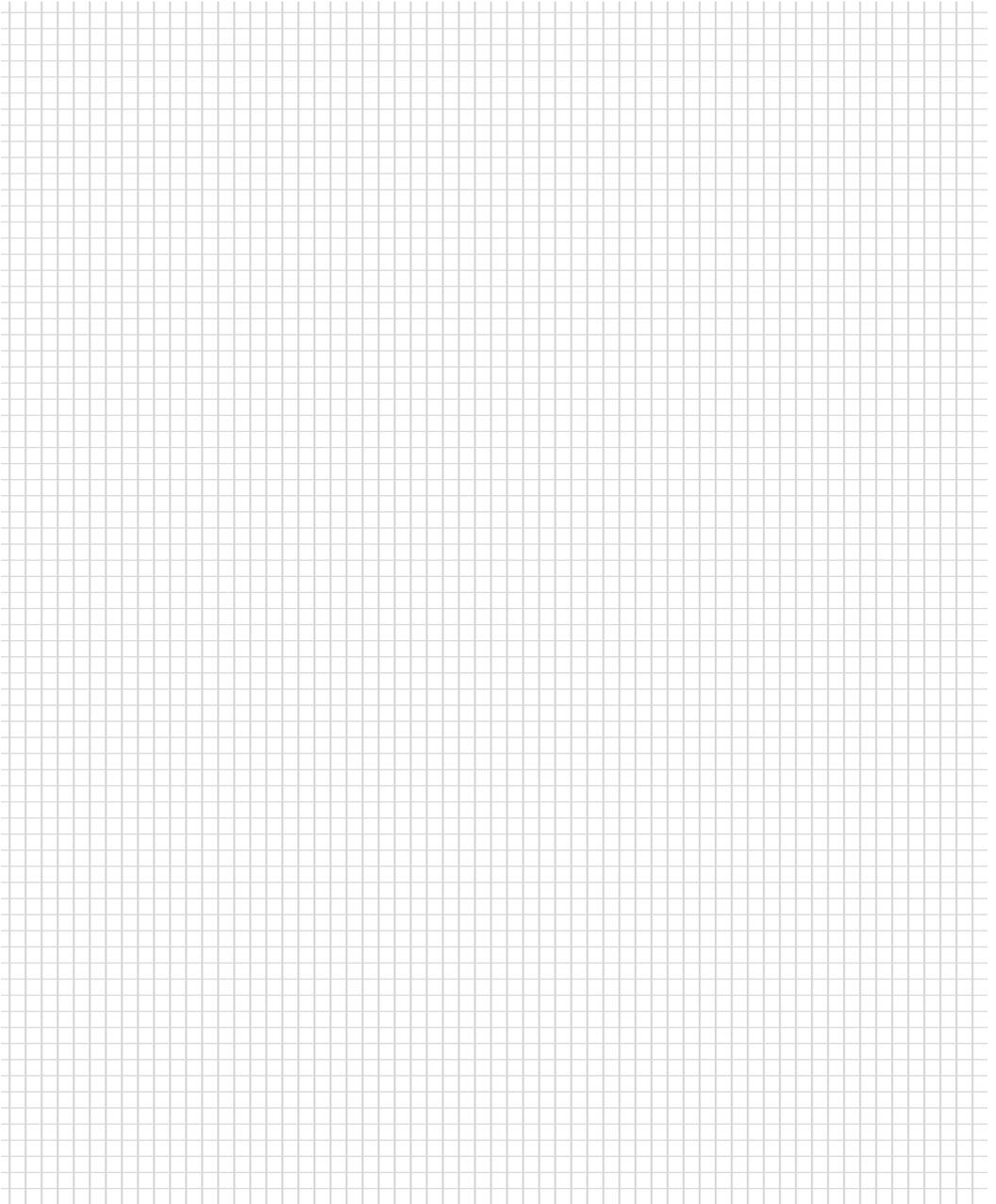2006/42/EC	Machinery Directive
2014/30/EC	Electromagnetic compatibility (EMC)
2014/35/EC	Low Voltage Directive
2012/19/EU	Disposal of waste electrical and electronic equipment
EN 61000-6-3	Interference emission
EN 61000-6-1	Interference immunity
EN 60335-1	Machine safety
EN 61010-1	Safety regulations for electrical measuring, control, and laboratory equipment
EN 61326-1	Electrical measuring, control and laboratory equipment

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