roeko Surgitip-endo

Handling Instructions	EN
-----------------------	----

Surgitip-endo is an aspirating tip specially designed to dry root canals. The innovative design of the suction tip allows for a multi-directional flexibility, so it can easily be introduced into hard-to-reach root canals without having to bend the Canal Tip. Aspiration is highly effective at all angles and visibility is not constricted.

Outer diameter canal tip = ISO 60
Inner diameter canal tip = 0,35 mm

- Please proceed as follows:**
- Arrange the flexible green tip so that it can be introduced into the canal vertically. Due to its multi-directional flexibility, this can be done without having to bend the tip.
 - The flexible tip of Surgitip-endo is made up of several parts, which are carefully assembled during production. However, to avoid over straining the joints, they should not be bent more than 90°. Otherwise they could come loose, with the risk of aspiration.
 - Use slight tapping movements during aspiration.
 - To prevent clogging the aspirator can be rinsed with water.
 - The extremely fine aspiration opening of the Canal Tip is not suitable for aspiration of tissue and/or larger particles. These can be removed as usual by irrigation and aspiration with the aid of Surgitip-micro or Surgitip.
 - For final drying and checking we recommend using a ROEKO/HYGENIC Paper Point.

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