PERMAPLAST BUILD UP

Stand:050418

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GB Instructions for use

Intended use

Dual Curing Flowable microhybrid Composite for Core Build-ups and Cementation of Posts.

M+W Permaplast Build Up is a dual curing radiopaque flowable hybrid composite based material for core-build-up and cementation of posts. Delivered in auto-mixing 1:1 MINIMIX-syringes it can be easily dispensed and applied directly. M+W Permaplast Build Up exhibits a short setting time with-out high heat generation. The dual cure properties enables the dentist, also to carry out cementations and core build ups in cases where a light cure cannot be guaranteed to be sufficient. M+W Permaplast Build Up meets the requirements of DIN EN ISO 4049.

Preparing the MINIMIX-Syringe

Remove the cap of the MINIMIX-syringe and throw it away (do not use it again!). It is replaced by a special 1:1 mixing cannula. Turn the cannula 90° until it locks in position. The material is now ready for application.

Note: Store used cartridge with fixed used mixing cannula. Discard the first 2-3 mm of the extruded material. This has to be done for each new mix. Store used syringe with fixed used mixing cannula in the dark. The working time (23°C) of M+W Permaplast Build Up in the self cure mode is 1:30 minutes from start of mixing.

1. Post Cementation

1.1. Isolation

Use of a rubber dam to isolate the tooth is strongly recommended.

1.2. Root Canal Preparation

Refer to directions of the selected post manufacturer. Remove all existing old restorations and decay from the tooth. Prepare and clean the root canal with e.g. sodium hypochlorite solution, rinse and remove excess solution from the canal with a soft paper tip. Condition and apply a bonding agent according to the manufacturer instructions. The total etch technique with etching gel and subsequent application of a dual cure bonding is recommended. Otherwise a self etching bonding can be used.

1.3. Post Cementation

Prepare the selected post according to manufacturer directions. M+W Permaplast Build Up is applied into the prepared root canal and onto the post. Seat the post carefully into the canal and maintain firm pressure until the post is seated. M+W Permaplast Build Up self-cures within 3:30 minutes. For post stabilization light cure the coronal part of the cemented post for 20 seconds. As soon as the M+W Permaplast Build Up has set proceed with the core-build-up procedure.

2. Core-Build-Up

2.1. Isolation

Use of a rubber dam to isolate the tooth is strongly recommended.

2.2. Cavity Preparation

Remove all existing old restorations and decay from the tooth. If necessary place any pins or posts. Refer to directions of the selected post manufacturer.

2.3. Pulp Protection Cavity floor of deep excavations should be covered with a thin layer of calcium hydroxide material.

2.4. Application of a Bonding Agent Condition and apply a bonding agent according to the manufacturer instructions. The total etch technique with etching gel and subsequent application of a dual cure bonding agent is recommended. Otherwise a self etching bonding can be used. **Note:** It is essential that the primed dentine and enamel surfaces are dry and contaminant free for the application of M+W Permaplast Build Up.

2.5. Application

Place the mixing cannula directly into the preparation and press out the paste. M+W Permaplast Build Up is automatically mixed when dispensed with slight and even pressure. Filling should occur from bottom upwards to prevent voids. To facilitate placement of M+W Permaplast Build Up place a matrix band around the prepared tooth. M+W Permaplast Build Up may be contoured by using a composite instrument. A flat-ended interproximal carver is recommended. Place M+W Permaplast Build Up directly into the preparation and allow the system to self cure for 3:30 minutes. After that the material should be light-cured (40 seconds). With this technique an optimum of physical properties will be obtained. An explorer can be used to test that the M+W Permaplast Build Up has completely set. Remove the matrix earlier than the preparation has set. Final crown preparation on the M+W Permaplast Build Up can be carried out by using crown preparation burs.

Additional Notes

Do not dilute the composite with resin. Do not use in conjunction with substances containing eugenol. Even the proximity of unsealed products containing eugenol is harmful. Therefore do not bring uncured composite into contact with products containing eugenol. Eugenol impairs the curing of the composite.

Storage

Do not store above 20 °C. Store unopened material in the refrigerator. Opened cartridges have been used up within 3 months. **Do not use after expiry date.**

Warranty

P.L. Superior Dental Material GmbH warrants this product will be free from defects in material and manufacture. P.L. Superior Dental Material GmbH makes no other warranties including any implied warranty of merchantability or fitness for a particular purpose. User is responsible for determining the suitability of the product for user's application. If this product is defective within the warranty period, your exclusive remedy and P.L. Superior Dental Material GmbH sole obligation shall be repair or replacement of the P.L. Superior Dental Material GmbH

Limitation of Liability

Except where prohibited by law, P.L. Superior Dental Material GmbH will not be liable for any loss or damage arising from this product, whether direct, indirect, special, incidental or consequential, regardless of the theory asserted, including warranty, contract, negligence or strict liability.

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Manufacturer: P.L. Superior Dental Materials GmbH · 22607 Hamburg · Germany
Vertrieb durch Müller & Weygandt GmbH · Reichardsweide 40 · D-63654 Büdingen · Tel.: +49 (0) 60 42 – 88 00 88