**VOCO****Traypurol** **Concentrated cleaning agent for
impression trays and instruments**

Traypurol removes residues of alginates, zinc oxide eugenol, carboxylate and phosphate cement from impression trays and instruments without using ultrasonic apparatus.

Traypurol is biodegradable and provides gentle cleaning action.

Instructions for use:**Performance features:**

The product's performance features satisfy the requirements of the intended use and the relevant product standards.

User:

Traypurol should only be applied by a professionally trained dental practitioner.

Use **Traypurol** in a 5 to 10 % solution (dilute approx. 50 - 100 ml **Traypurol** in 1 l water). For persistent residues, **Traypurol** can also be used in a more concentrated solution. Warm water accelerates the cleaning process.

Pre-clean the trays and instruments and place them in the **Traypurol** solution. An upright position of the impression trays in the solution will enhance the cleaning process.

Residues are softened in the **Traypurol** solution within a few hours and can then be easily rinsed off under running water or removed with a brush. If possible leave impression trays and instruments soaking overnight in the solution.

Traypurol is suitable for all non-corroding metals and plastics. Renew the solution at least after 2 - 3 days. **Traypurol** is also very suitable for use in ultrasonic apparatus.

Warnings/precautionary measures:

- Safety glasses and gloves should be worn while using this product.
- After eye contact rinse intensively with water and contact physician.

Storage:

Store at temperatures of 4 °C to 23 °C. Do not use after expiry date.

Disposal:

Dispose of the product according to local regulations.

Reporting obligation:

Serious events such as death, temporary or permanent serious deterioration of a patient's, user's or other person's condition and a serious risk to public health that arise or could have arisen in association with the use of **Traypurol** must be reported to VOCO GmbH and the responsible authority.

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VOCO GmbH Phone +49 (4721) 719-0