SUPERLITE POWER PEN/ COLOR PEN

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Seite 2-10

(GB) INSTRUCTIONS FOR USE

Page 11-19

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Page 20-28

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Pagina 29-37

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Strana 38-46

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Oldal 47-55

Standt:171219

 BG LIGHT LTD · 155, Vasil Aprilov blvd. · BGR - 4027 Plovdiv · tel.: +359 32 644089

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Instructions for use

Attention!

Read the operating instructions carefully before installation and operation of the device.

Contents:

- I. Description and mode of function of the M+W Superlite Power Pen/Color Pen
- II. Technical data
- III. Accessories and spare parts
- IV. Safety precautions
- V. Preparation and sequence of operation
- VI. Care and maintenance
- VII. Warranty conditions
- VIII. Symbols
- IX. Date of purchase / serial number M+W Superlite Power Pen/Color Pen (to be filled in by purchaser)

I. Description and mode of function M+W Superlite Power Pen/Color Pen

Cordless LED light polymerisation lamp for intraoral hardening of light-curing, dental materials. The wavelength of the emitted light lies in the range 410 nm to 490 nm. The M+W Superlite Power Pen/Color Pen is manufactured taking into account Directive 93/42 EEC and Directive 2007/47/EC.


II. Technical data

1. Operating voltage: 3.7 V Li-ion rechargeable battery (800 mAh)
2. Power consumption: mains adaptor – 110-240 VAC/50-60 Hz, 0.2-0.1 A Output: 5 VDC/1.2A micro USB socket
3. Dimensions: diameter of handpiece approx. 25 mm, length of handpiece without light conductor 148 mm
4. Weight of handpiece: 105 g
5. Light intensity: up to 1.100 mw/cm²
6. Exposure duration: soft start mode, settings 10 sec./ 20 sec. / 30 sec., continuous operation up to max. 10 minutes
7. Wavelength range: 410 nm to 490 nm, blue emittance
8. Light source: 4 powerful LED crystals in module
9. Charging time: 6-8 hours for full charge and 3-4 hours to 80% of full charge.
10. Operating duration: 2.000 seconds for fully charged battery
11. Signal: acoustic signal every 10 seconds and colour indicator ring on handpiece which indicates the condition of the device. See V./section 3

III. Accessories and spare parts

1. Charging cable
2. Handpiece
3. Li-ion rechargeable battery (already included in handpiece)
4. Light conductor 8mm
5. Light protection shield
6. Light protection cap for posterior dental region
7. Holder for handpiece (for M+W Superlite Color Pen the holder is available as an option)

Instructions for use

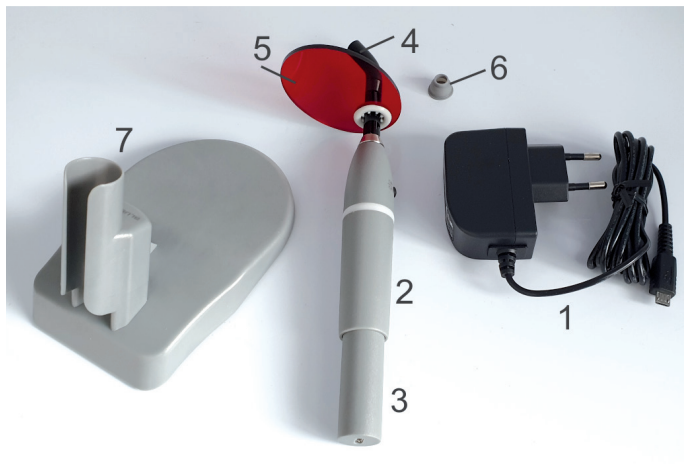IV. Safety precautions

As with all technical devices, the proper function and safe operation of this device depend on the user's compliance with the standard safety procedures as well as the specific safety recommendations described in these operating instructions.

1. The device must be used in strict accordance with the Operations guide. The manufacturer accepts no liability for any damage resulting from the use of this device for any other purposes.

2. Before start-up of the device make sure that the operating voltage indicated on the device label corresponds to the mains voltage. Operation of the device at a different voltage may damage the device.

Electrical safety is ensured by Protection class II against electrical current damages according to EN 60601-1. Use of a safety plug or earthing of the device is not necessary.

The M+W Superlite Power Pen/Color Pen should be operated in a room under the following conditions:

- temperature: $+10^{\circ}$ – $+40^{\circ}$ °C
- relative air humidity: 30 – 75 %
- atmospheric pressure: 700 – 1060 hPa
- no chemically active or inflammable materials in the vicinity

3. Irradiation of the eyes bears certain risk for their damage. Do not direct the light directly at the eyes! Exposure must be restricted to the working area.

ATTENTION! The emitted light is in the wavelength range 410-490 nanometers. Use of protection glasses which effectively block the blue spectral range is recommended!

4. The device may only be used by or on the following persons after consultation with a physician: persons with photobiological reactions; persons currently taking photosensitive medication; persons with retinal diseases, etc.

5. Solvents, flammable liquids and powerful heat sources must not be kept near the device as exposure to one of these factors may damage the plastic parts of the device.

6. Cleaning agents should not penetrate the device as this may short-circuit the device or cause potentially dangerous malfunction.

7. Only factory-authorized technicians are authorized to open and repair the device.

8. Defective parts may only be replaced with original spare parts for the

Warranty for the device does not cover any damage resulting from the use on non-original replacement parts. The device or any of its parts should not be disassembled while it is still connected to the mains!

9.  ATTENTION FRAGILE! Be extremely careful during transportation, storage and handling of the device!

10.  Pursuant to Directive 2002/96/EU, this symbol shows that the product must not be disposed of as urban waste at the end of its operating life. Devices removed from service must be taken to centres specialised in separate collection of electric and electronic equipment. Correct disposal of equipment which is no longer used, prevents negative consequences for the environment and human health!

11. Prior to every application, the light conductor, the glare protection shield and the light protection cap should be prepared according to the recommendations in VI. Care and maintenance.

12. Heat generation

As with all high performance lamps, the high light intensity involves heat generation. Irreversible damage can occur in the region near the pulp or soft tissues during prolonged exposure. Therefore, the prescribed exposure times in the region near the pulp (adhesive 10 seconds) are to be observed. Uninterrupted exposure times of more than 20 seconds on the same tooth surface as well as direct contact with the gingiva, oral mucosa or skin are to be avoided under all circumstances. In the case of indirect restorations, work should be performed with intermittent intervals of 20 seconds or via external cooling with air flow.

V. Preparation and sequence of operation

1. Remove M+W Superlite Power Pen/Color Pen from the packaging.




2. Connect power adaptor to mains to charge the battery.

Attention! The device should be placed at least 1 metre away from any heat source.

3. Prior to being used for the first time, the light conductor, the glare protection shield and the light protection cap should be prepared according to the recommendations in VI. Care and maintenance.


Then re-attach light protection shield or light protection cap.

The light protection cap is suitable for polymerisation in the posterior dental region, the light protection shield is suitable for the anterior dental region.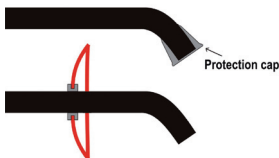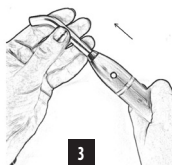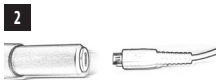

4. Colour ring indicator in the handpiece

- 4.1 Lamp is inactive – the colour ring indicator does not display any colours
- 4.2 Charging of battery – the colour ring indicator flashes blue
- 4.3 Batteries are fully charged – the colour ring indicator shines blue permanently
- 4.4 During polymerisation – the colour ring indicator shines green
- 4.5 Protection against overheating – the colour ring indicator flashes yellow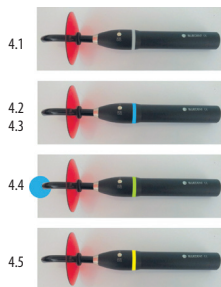

5. Charging and replacing the battery

When emitted light is blinking, this means the battery is 85% discharged. It can be used for a short time to finish the curing procedure and must be connected to charger after that.

The M+W Superlite Power Pen/Color Pen is fitted with an electronic charge control which prevents overcharging the battery and thus ensures a long service life of the battery.




Only use the supplied mains plug for charging the M+W Superlite Power Pen/Color Pen!

The practical mains plug allows use of the M+W Superlite Power Pen/Color Pen during charging. This avoids long waiting periods for charging.

To replace the battery, unscrew the lower section of the M+W Superlite Power Pen/Color Pen as shown in the illustration.

Loosen the white plug connection and remove the battery from the device.
Reinsert the new battery in the same manner.

Only use original M+W Superlite Power Pen/Color Pen batteries, do not use with other batteries.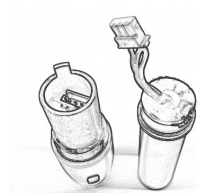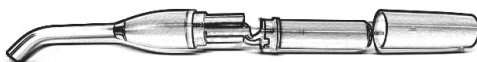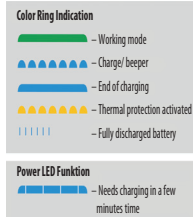

6. How to use

Press the ON/OFF switch.

By pressing once, exposure is started for 10 seconds.

The exposure time can be extended to 30 seconds by depressing the ON/OFF button repeatedly several times.

Press 2x = 20 seconds / press 3x = 30 seconds.

Hold the light conductor as close as possible to the region to be cured (4-7 mm). The light conductor must not contact the dental material for the first few seconds. Once the first layer is sufficiently cured, the light conductor may come into contact with the dental composite.


The device is equipped with overheating protection, which is activated if the temperature reaches 50°C. When overheating protection is activated, the colour indicator ring will flash yellow for the next 60 sec. and is then deactivated until the temperature of the handpiece reaches 35°C.

For safety reasons never charge the device unattended. Therefore, please remove the plug when leaving the dental practice.

VI. Care and maintenance

1. Remove hardened plastic residues from light conductor. Do not use metallic, pointed or sharp objects for this purpose.

2. Disinfect the handpiece with a soft cloth and disinfectant solution. Please do not spray, only wipe. Liquids should not penetrate into the interior of the handpiece. Do not use abrasive cleaning agents. We recommend M+W alcohol-free disinfection spray. Moisten the cloth and carefully wipe the handpiece.

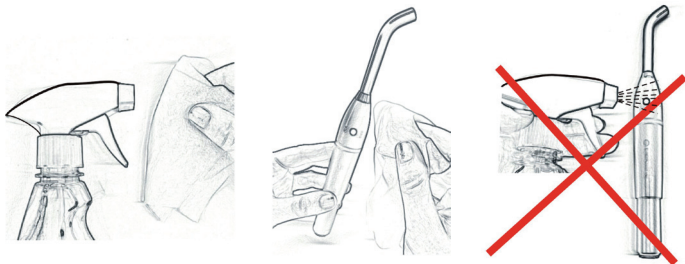Remove and remaining composite residue from the light conductor.

Manual and machine preparation of light conductor, glare protection shield and light protection cap:

- do not use abrasive cleaning agents or wire brushes.

Manual cleaning

Accessories: brushes with soft bristles, lint-free cloths, alkaline cleaning agents, i.e. M+W Instrument bath AF, distilled water

Step	Duration	Description
1	1 min.	Rinse used parts under cold water and remove coarse, visible contamination immediately.
2		Now place pre-cleaned parts in alkaline cleaning agent. Please ensure complete wetting. Observe application concentration and exposure time of the disinfectant agent.
3	1 min.	Then rinse with clean water. Should visible contamination persist, please repeat Steps 1-3 until all visible contamination has been removed.
4	1 min.	Then rinse parts with distilled water and dry.
5		Final visual inspection for contamination. If necessary, repeat Steps 1-6.

Machine preparation using a CDD (only light conductor and light protection cap) Preparation: for pre-cleaning see Step 1 for manual cleaning. Visual inspection. Should visible contamination persist, pre-clean again manually until the contamination is removed. Then place in the thermal disinfectant. Ensure that the light conductors cannot come into contact with other objects during preparation (risk of breakage!)

Start the cleaning programme of the thermal disinfectant:

Programme	Duration	Description
Pre-cleaning	2 Min.	
Cleaning	2 Min.	
Neutralisation	5 Min.	
Disinfection	5 Min.	90° C
Rinsing	2 Min.	
Drying	40 Min.	90° C

After cleaning is completed, check the parts carefully for damage, scratches, etc. Dispose of damaged parts.

Sterilisation of lighting conductor and light protection cap:

Pack parts in suitable packaging material (according to DIN 58 953/7 or EN 868) Then sterilise in autoclave at 134°C

VII. Warranty conditions

1. Warranty period of the M+W Superlite Power Pen/Color Pen is 24 months from the date of purchase. If date of purchase is not received in written form, warranty starts at date of production.
2. During the warranty period any defective parts (LED module and battery - 6 months warranty) are replaced free of charge by the manufacturer.
3. If the device is damaged during the guarantee period due to incorrect operation (mechanical, chemical, thermal or electrical damage), improper use, storage or any other reasons where the user is at fault, the repairs are to be paid by the user.

If any damage is noticed to the electric cables, the device must be sent immediately to the manufacturer. The device must not be used if cables are damaged. If liquids, such as water and solvents, aggressive or flammable substances and their vapours penetrate or are poured on the device, or if the device is wiped with same, or if insects or rodents have entered the device, it must be sent immediately to the manufacturer. In such cases warranty claims are inadmissible. Damages and claims relating to irradiation are not accepted if resulting from electric shocks, thunder storms, non-compliance with measures for electric safety or insufficient protection of patients, personnel or third parties.

The manufacturer does not owe compensation for lost profits in cases of defects or malfunction of the device, regardless of the cause.

No claims and damages are accepted due to inadequate safety measures and lacking security during transportation, unpacking, changes in position and work location, and the storage of the device. Warranty is null and void if any of the above circumstances occur.

If a dispute arises on the application and interpretation of these operating instructions, the applicable laws in Germany shall apply. Place of jurisdiction is the court in Friedberg.

4. This warranty expires if repairs or servicing of the device are performed by an unauthorized party or if other than original parts of the M+W Superlite Power Pen/Color Pen are used.

5. The warranty services are performed at the manufacturer's service centre at the following address:

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VIII. Symbols

	Manufacturer M+W Dental Müller & Weygandt GmbH Reichardsweide 40, 63654 Büdingen
	Pursuant to Directive 2002/96/EU, this symbol indicates that a product must not be disposed of as urban waste at the end of its operating life.
	Pay attention to the instructions accompanied by this symbol.
	Attention fragile!
	Product with CE label.
	Applied part type B.
	The device is equipped with double electric insulation (Class II).
Ω	Ohm (unit of electric resistance)
s	Second (unit of time)
W	Watt (unit of power)
Hz	Hertz (unit of frequency)
mm	Millimetre (unit of length)
A	Ampere (unit of electrical current)
V	Volt (unit of electrical voltage)
Pa	Pascal (unit of atmospheric pressure)
°C	Degrees Centigrade (unit of temperature)
g	Gram (unit of weight)
SN	Serial number
LOT	Lot
QK	Quality control

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M+W Dental® Vertrieb durch Müller & Weygandt GmbH · Reichardsweide 40