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Product description

GumQuick: Flexible gingiva mask material for the C&B technique
GumQuick implant: Hard/flexible gingiva mask material for the implantology
Both GumQuick materials are advanced addition-curing vinyl polysiloxanes (VPS) designed for creating flexible gingiva masks on master models.

Indication for use

GumQuick: Gingiva mask made in the dental laboratory for C&B. GumQuick attains a final hardness of 60 Shore A and reproduces the natural proportions in combination with the opaque gingiva color. Therefore, unphysiological pressure onto the gingiva parts can already be avoided during the wax-up.
GumQuick implant: Gingiva mask made in the dental laboratory for highly precise implant works.
GumQuick implant has been developed as special gingiva mask material for the implant technique. The high final hardness of 75 Shore A and the low flexibility of the material guarantee a safe and easy modellation of the secondary parts taking into consideration the natural course of the gingiva.

Processing

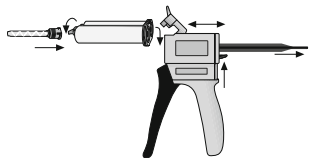
GumQuick (Indirect Method)

Apply a matrix of hard, kneadable silicone (e.g., Laborsil pro) onto the dies of the unsawn model, which should already include the gingiva mask. After curing, trim the silicone so the tops of the dies are exposed (approx. 1 mm). The model can then be sawn and the gingiva parts ground as usual. Afterward, verify that the matrix is seated correctly. To guarantee a flawless surface, apply a separator to the contact areas between the silicone and GumQuick.

Apply GumQuick directly from the cartridge onto the model and into the matrix. Position the matrix on the model so that excess material can flow out through the openings at the top of the die. Ideally, the gingiva mask should cure under pressure. After approximately 5:30 minutes, remove the matrix. The mask can then be trimmed using scissors, a scalpel, or rotating instruments.

GumQuick Implant (Direct Method)

First, apply DuoSep to the contact surface between the impression material and the GumQuick Implant to ensure a flawless surface.
Caution: To prevent damage to the gingiva mask surface when using GumQuick Implant with polyether impressions, DuoSep must be used!
Allow polyether impressions to ventilate for at least 3 hours before producing the mask. Afterward, apply GumQuick Implant directly from the cartridge into the impression around the transfer posts. Thanks to its excellent flow characteristics, the material will securely surround the abutments. After approximately 5:30 minutes, the material is cured, and the impression can be poured with plaster. The gingiva mask can then be easily trimmed with scissors, scalpels, or rotating instruments.

Caution

- This product is exclusively intended for the applications mentioned above and must only be used by qualified and trained professionals.
- The polymerization (setting) of cross-linked silicones can be inhibited by latex gloves, hand creams, plastics, and cleaning agents. We recommend wearing commercially available polyethylene (PE) gloves.
- Setting times may increase at lower temperatures and decrease at higher temperatures.
- Wear appropriate protective gear. Cured material is chemically resistant; marks on clothing cannot be removed.
- Always bleed (pre-convey) the material before attaching a new mixing tip.
- Never attempt to dispense material through a clogged or cured mixing tip.
- After use, leave the mixing tip on the double cartridge to serve as a seal.

Keep out of reach of children!

Batch number/Expiry date

The batch number and expiration date are located on both the outer packaging and the product. Please always provide the batch number when submitting a complaint. Do not use the product after the expiration date.

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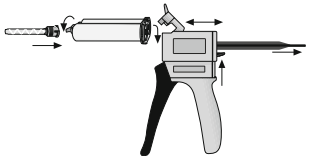
This product was developed specifically for dental use and must be used in accordance with the instructions for use. The manufacturer assumes no liability for damages resulting from improper use or applications not specified in the instructions. Furthermore, the user is responsible

for verifying the product's suitability and intended use for their specific purpose prior to application, especially if the intended use is not explicitly mentioned in the documentation.

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