VOCO**Individuo Lux****GB****blue opaque and blue transparent****Instructions for use****Product description:**

Individuo Lux blue opaque and **Individuo Lux blue transparent** are opaque and respectively, transparent light-curing resins in the form of contoured plates for maxillae and mandibles. They are designed for the fabrication of individual trays for use in crown, bridge and total prosthetic techniques, as well as for the fabrication of bases for bite patterns and odontograms. They are ready for use and easy to handle without any odour nuisance. **Individuo Lux** can be worked for a long time under artificial and day light, and cures quickly under halogen or UV light. The individual trays can be used directly after fabrication. **Individuo Lux** has a pleasant peppermint flavour.

Method of Use:

Allow **Individuo Lux** to reach room temperature before use. Mark margins for the individual tray on the plaster model. If necessary, use a space maintainer. If a space maintainer is not necessary, insulate the plaster model.

Remove protective foil and mould plate onto the model, shape and cut the desired rim. The transparency of **Individuo Lux blue transparent** allows very exact modelling. The excess material can easily be used for making a handle.

Individuo Lux is then polymerised on the model in a standard light unit (halogen or UV light). Depending on the thickness of the contoured tray, polymerise the reverse side for 3 - 5 min. A longer polymerisation time will improve the physical properties of the material.

Therefore, polymerisation time should be prolonged, if necessary.

The custom tray is then immediately ready to be used or worked. A slight inhibition layer, which may be caused by the oxygen in the air, can be removed with a tissue moistened with alcohol.

Storage:

Store at temperatures of 4 °C to 23 °C. Store in cool conditions and protected from light.

Precautions/indications:

Exposure that is too short or not sufficiently intense will lead to a loss in material strength. **Individuo Lux** contains Bis-GMA, UDMA, TEGDMA, peppermint oil, BHT. **Individuo Lux** should not be used in case of known hypersensitivities (allergies) to any of these ingredients.

Do not use after expiry date.

Our preparations have been developed for use in dentistry. As far as the application of the products delivered by us is concerned, our verbal and/or written information has been given to the best of our knowledge and without obligation. Our information and/or advice do not relieve you from examining the materials delivered by us as to their suitability for the intended purposes of application. As the application of our preparations is beyond our control, the user is fully responsible for the application. Of course, we guarantee the quality of our preparations in accordance with the existing standards and corresponding to the conditions as stipulated in our general terms of sale and delivery.

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VC 60 AA2400 E1 1013 99 © by VOCO
VOCO GmbH