EASYblank wax

CAD/CAM Wax Blank

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Produktbeschreibung

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ENGLISH | INSTRUCTIONS FOR USE

Product description

Milling blank made from high-quality synthetic components for the production of milled objects which can be burned-out without leaving a residue.

Indication

Ideal for the production of crown and bridges in casting techniques and pressing techniques in ceramic systems. The optimized formula guarantees a very high edge stability and very clean chip formation. For use in open milling systems Only to be used by trained dental personnel.

Contraindication

Application in the mouth or on the body is not permitted. This material is not designed for manual waxing techniques.

Special hazard warnings

Avoid contact of hot molten wax with skin or eyes.

Processing

- The EASY blank must be secured in the milling machine according to the relevant device manufacturer's instructions.
- Use milling cutters suitable for milling wax blanks
- After processing, separate the milled objects from the blank using fine-toothed carbide cutters or separating discs.
- Sprue and invest the wax patterns according to the framework design and further processing technique (casting / pressing). In this case, follow the investment material manufacturer's instructions.

Scope of Delivery

Ø 98,5 mm × 14 mm, blank with step / shoulder, blue –
REF: 770 0014

Ø 98,5 mm × 20 mm, blank with step / shoulder, blue –
REF: 770 0020

Storage

- Maximum storage temperature 35 °C
- Protect against direct sunlight
- Protect against impact and vibration

Disclaimer

This product has been developed and is intended exclusively for professional use by trained dental professionals. The above information on processing is to be regarded as non-binding information. The use and processing take place under the responsibility of the user. Renfert assumes no liability for damages that arise from improper handling or processing.

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