3M ESPE**KetacTM Silver
AplicapTM/MaxicapTM****(en) Silver-reinforced Glass Ionomer Restorative Material – Radiopaque****(de) Silberverstärktes Glasionomer Füllungsmaterial – Röntgensichtbar****(fr) Matériaux d'obturation en verre ionomère à renfort d'argent – Radiopaque****(it) Materiale per otturazioni vetroso-ionomero rinforzato – Radiopaco****(es) Material de obturación de vidrio ionómero reforzado con plata – Radiopaco****(pt) Material de obturação de vidro ionómero reforçado com prata – Radiopaco****(nl) Met zilver versterkt glasionarioer vulmateriaal – Radiopak****(el) Υλικό εμπράξεων από υαλοϊονομέρη κονία εντοχυμένη με όργυρο – Ακτινούσκερο****(sv) Silverförstärkt glasjonomer fyllningsmaterial – Röntgenpaket****(fi) Hopeavahvisteinen lasi-ionomeeri-täytämateriaali – Röntgenpaaikki****(da) Sølvforsterket glasionomer-fyllningsmateriale – Radiopaque****(no) Sølvforsterket glassionomerfyllingsmateriale – Med røntgenkontrast****Instructions for Use
Gebrauchsinformation
Mode d'emploi
Informazioni per l'uso
Información de uso
Instruções de uso****Gebruiksinfo
Οδηγίες χρήσης
Bruksanvisning
Käyttöinfo
Brugsanvisning
Bruksinstruksjon**

010009645/01

**3M Deutschland GmbH
Dental Products
Carl-Schurz-Str. 1
41453 Neuss – Germany****3M ESPE Customer Care/MSDS Information:
U.S.A. 1-800-634-2249 and Canada 1-888-363-3685.
3M, ESPE, Aplicap, CapMix, Ketac, Maxicap and RotoMix are trademarks
of 3M or 3M Deutschland GmbH. Used under license in Canada.
© 2020, 3M. All rights reserved.****ENGLISH****Product Description**
KetacTM Silver AplicapTM and MaxicapTM are silver-reinforced glass ionomer cements in capsule form intended for direct application. Their chemical adhesion on enamel and dentine provides a tight marginal seal without the need for extensive preparation. The material can be applied without lining, releases fluoride ions, and is radiopaque. The minimum amount which can be dispensed from one Aplicap capsule is 0.13 ml; the minimum amount which can be dispensed from one Maxicap capsule is 0.45 ml.**☞ The instructions for Use must be kept for the duration of product use.****Indications**

- Amalgam base linings
- Core build-ups
- Uni-planar fillings and cervical fillings in cases where the esthetic effect is not of primary consideration
- Primary tooth fillings

Precautionary Measures
3M MSDSs can be obtained from www.mmm.com or contact your local subsidiary.**Preparation**
► Only remove decayed tooth substance. Undercuts are not necessary.**Pulp Protection**
► In order to prevent pulpal irritation, cover areas in close proximity to the pulp with local application of a calcium hydroxide material.**Conditioning****►** For optimal chemical bonding to the teeth, the smear layer produced in the preparatory steps must be carefully removed by applying Ketac Conditioner to the prepared surfaces and allowing to react for 10 sec.**►** Then rinse with copious amounts of water.**►** Blow the cavity dry in only 2–3 short intervals with water- and oil-free air or dab dry with cotton pellets. Do not overdry! **The cavity should be just so dry that the surface has a matt shiny appearance.** Excessive drying can result in post-operative sensitivity after filling.**►** Avoid further contamination.**Activation and Mixing****►** Place the activator on a sturdy surface and insert the Aplicap capsule in the Aplicap Activator, or the Maxicap capsule in the Maxicap Activator.**►** Using the ball of your hands, depress the activator lever firmly and completely to the stop and hold it down for 2 to 4 seconds.**►** Firmly and completely depressing the lever to the stop and holding it in this position is the only way to ensure that all of the liquid is pressed into the powder. If inadequate pressure is applied or if the lever is not depressed completely, it is possible that too little liquid will be mixed with the powder, increasing the viscosity and changing the product characteristics.**►** Mix immediately using a high-speed mixer (approximately 4300 oscillations per minute), e.g., CapMix, or the RotoMix rotary mixer; please refer to Times.**Application**
► Apply with the petalular applicator which fits the capsule.**►** During the application and the setting phase, the working area should be protected from water and saliva. Avoid desiccation of the fresh filling, e.g. by a strong air stream.**►** When using the material as a build-up under amalgam, the cavity has to be filled in such a way that the dentin is completely covered.**Attention**
• Pauses between activation, mixing, and application must definitely be avoided; otherwise, as soon as the material starts to set, removal from the capsule will be rendered difficult or even impossible.**• Ketac Silver Aplicap and Maxicap adhere to metal instruments and should therefore be removed with cold water before setting.****Times**
The following times apply to a room temperature of 23°C/73°F:

	min:sec
Activation	0:02
Mixing in RotoMix	0:08
Mixing in high-speed mixer, e.g., CapMix	0:10
Working (from beginning of mix)	2:00
Setting (from beginning of mix)	5:00

Working and setting times are accelerated at room temperatures above 23°C/73°F. Exceeding the working time will lead to loss of adhesion to enamel and dentine.**Shorter mixing times will prolong the working and setting times whereas longer mixing times will shorten them.****– Aplicap capsules: on high-speed mixers, the mixing time can be set from 8–15 sec and on the RotoMix from 7–12 sec.****– Maxicap capsules: on high-speed mixers, the mixing time can be reduced to 8 sec and on the RotoMix to 7 sec.****Filling Protection**
Glass ionomers set in two phases and are susceptible to moisture during the first setting phase. Subsequently, dehydration may occur, which can be prevented by applying vaseline and/or cotton rolls immediately after removal of strips or matrix.**►** Apply vaseline to all exposed areas of the glass ionomer cement.**Finishing**

- Remove excess with a sharp blade.
- Finishing should be carried out using rotating instruments 5 minutes after start of mixing at the earliest.
- For fine finishing, use discs or rubber cups and water-cooling.
- If requested, apply vaseline again for surface sealing.

Storage and Shelf Life

Store the product at 15–25°C/59–77°F.

Do not use after the expiry date.

Customer Information

No person is authorized to provide any information which deviates from the information provided in this instruction sheet.

Warranty

3M Deutschland GmbH warrants this product will be free from defects in material and manufacture. 3M Deutschland GmbH MAKES NO OTHER WARRANTIES INCLUDING ANY IMPLIED WARRANTY OF MERCHANTABILITY OR FITNESS FOR A PARTICULAR PURPOSE. User is responsible for determining the suitability of the product for user's application. If the product is defective within the warranty period, your exclusive remedy and 3M Deutschland GmbH's sole obligation shall be repair or replacement of the 3M Deutschland GmbH product.

Limitation of Liability

Except where prohibited by law, 3M Deutschland GmbH will not be liable for any loss or damage arising from this product, whether direct, indirect, special, incidental or consequential, regardless of the theory asserted, including warranty, contract, negligence or strict liability.

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