VOCO Aqua Meron

GB

CE 0482

In accordance with DIN EN ISO 9917-1

Instructions for use

Product description:

Aqua Meron is a water-mixable glass ionomer cement for luting.

Its outstanding features include low solubility in the mouth and a low level of acidity.

Fields of application:

For attaching crowns, bridges, inlays, onlays, posts and orthodontic bands, as well as for the conventional luting of high-strength full ceramic restorations as for example ceramics of silicates, zirconium oxide, aluminium oxide.

Method of Use:


Clean enamel, dentine and metal surfaces carefully and dry thoroughly.

Attention! **Aqua Meron** must be mixed only with water! Shake powder well before use!

Aqua Meron has to be applied at room temperature (15 °C - 25 °C). In case of higher temperatures it is recommended to use a cooled glass plate.

Place powder and water on a glass plate or mixing pad and work the powder portion by portion into the water. Use 2 drops of water for 1 level measuring scoop of powder (mixing ratio powder : water = 3.3 - 3.8 g : 1).

When dispensing the water hold the dropper bottle vertically, making sure that the metal canula protrudes approx. 2 - 3 mm above the top.

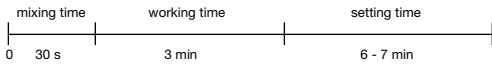The mixing time is approx. 30 s. **Aqua Meron** can be worked at room temperature for approx. 3 min after commencement of mixing and sets in the mouth within 3 - 4 min. Higher temperatures will shorten the working time and lower temperatures will prolong it.

In case of narrow bridges or crowns it is important to ensure that the surplus of **Aqua Meron** is not too great, nor its viscosity too high, since glass ionomer cements have specific flow properties.

Aqua Meron has reached the right consistency when it can be applied to the tooth cores and inside the crown with a stiff brush. During application and setting time prevent contact with water and saliva.

Supplementary indications:

Rinse metal instruments with cold water immediately after use and before any residual cement has time to set. After removal of orthodontic bands, cement residues can be removed gently and rapidly by means of an ultrasonic scaler.

Precautions:

Do not apply large excess amounts of cement to prevent irritations of the pulp.

Close powder container carefully after use in order to avoid infiltration of humidity and to preserve the high strength.

Avoid contact with mucosa and eyes!

Aqua Meron contains Na-Ca fluorosilicate glass and polyacrylic acid. **Aqua Meron** should not be used in case of known hypersensitivities (allergies) to any of these ingredients.

Do not use insulating materials either in order to maintain full adhesion to the dentine.

Storage information:

Close the **Aqua Meron** powder bottle tightly and store at temperatures of 4 °C to 23 °C. Protect from humidity.

Do not use after expiry date.

Our preparations have been developed for use in dentistry. As far as the application of the products delivered by us is concerned, our verbal and/or written information has been given to the best of our knowledge and without obligation. Our information and/or advice do not relieve you from examining the materials delivered by us as to their suitability for the intended purposes of application. As the application of our preparations is beyond our control, the user is fully responsible for the application. Of course, we guarantee the quality of our preparations in accordance with the existing standards and corresponding to the conditions as stipulated in our general terms of sale and delivery.

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VC 60 AA1172 E1 0114 99 © by VOCO
VOCO GmbH