Product description:

Meron Plus is a resin-reinforced, chemically curing glass ionomer cement (glaspolyalkenoate cement) for permanent luting. The material's characteristics are easy handling, low solubility in the mouth and high self-adhesion.

Fields of application:


- Luting of:
- porcelain-fused-to-metal crowns, bridges on hard dental tissue and cores rebuilt with amalgam, composite or glass ionomer cements
 - metal inlays, onlays, crowns and bridges
 - posts
 - full porcelain crowns manufactured from silicate-only, alumina-only or zirconia-only cores
 - orthodontic appliances

Method of use:

Thoroughly clean and dry enamel, dentine and cavity surfaces. Clean cores before luting (e. g. pumice, water), rinse thoroughly and dry with a gentle air stream or cotton pellets (Do not overdry!).

Shake the powder and the liquid bottle thoroughly for at least 3 s prior to dispensing.

The working temperature should be 15°C - 23°C. Mix one level spoon of powder to two drops of liquid. Place powder and liquid onto a glass mixing plate or a suitable mixing block. Hold the bottle vertically when dispensing the liquid. Mix powder and liquid into a homogeneous paste with an abrasion-resistant spatula.

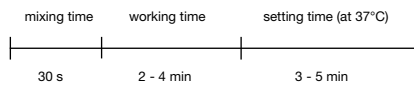Application:

Luting of restorative pieces:

Apply the material with a suitable instrument (spatula, brush) onto the preparation and/or into the restorative piece. Then place the piece in situ and let it set under slight pressure. **Meron Plus** has a working time of approx. 2 - 4 min at room temperature and sets in the mouth in 3 - 5 min. Higher temperatures shorten the working time, lower temperatures prolong it. After approx. 4 minutes the material reaches a gel-like consistency so that excess can easily be removed with a probe.

Luting of posts:

Apply **Meron Plus** only to the post and then seat the post in the root canal. Make the post vibrate slightly while seating it to avoid enclosing air bubbles. Do not apply the cement into the canal with a lentulo since this could speed up the setting excessively.

Indications and precautions:

Avoid contact of **Meron Plus** with eugenol, thymol or other substances which inhibit the setting of **Meron Plus**. Attach temporary pieces with eugenol-free temporary cements. Before final luting, remove completely all residues of varnishes and other substances (e. g. isolating agents) from the hard substance as they may impair adhesion. Ensure sufficient residual dentine thickness to prevent pulp irritation. If necessary, protect the pulp or build up the core.

Rinse instruments immediately with cold water before the cement residues have set. After removal of orthodontic bands, remove cement residues quickly and gently with an ultrasonic scaler. Avoid contact of **Meron Plus** with soft tissue.

Meron Plus contains polyacrylic acid, peroxide, amines, fluorosilicate, HEMA; glycerinedimethacrylate and diurethanedimethacrylate. Do not use **Meron Plus** in cases of known hypersensitivities to these ingredients.

In case of eye contact rinse thoroughly with water and contact your eye specialist.

Storage information:

Close containers of powder and liquid tightly immediately after use. Humidity can cause lumpiness and reduce solidity. Store in dry conditions at 4°C - 23°C. Do not use after the expiry date.

Our preparations have been developed for use in dentistry. As far as the application of the products delivered by us is concerned, our verbal and/or written information has been given to the best of our knowledge and without obligation. Our information and/or advice do not relieve you from examining the materials delivered by us as to their suitability for the intended purposes of application. As the application of our preparations is beyond our control, the user is fully responsible for the application. Of course, we guarantee the quality of our preparations in accordance with the existing standards and corresponding to the conditions as stipulated in our general terms of sale and delivery.

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VC 60 AA 1731 E1 0719 99 © by VOCO
VOCO GmbH