SEPTOPRINT KONZENTRAT

REF 100 725


**Inhalt/Content/Contenu/Zawartość/
Obsah/Tartalma:**
e 1 l

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- Ⓒ Concentrate for the disinfection of impressions
- Ⓕ Concentré de désinfection de tous les matériaux d'empreinte dentaires.
- Ⓖ Konzentrat do dezynfekcji wycisków stomatologicznych i prac protetycznych
- Ⓒ Kocentrovaná dezinfekce na otisky
- Ⓗ Lenyomatfertőtlenítő koncentrátum

Gefahr/Danger/
Niebezpieczeństwo/
Nebezpečný/Vezely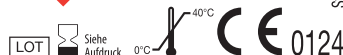

Stand: 09/18

PRISMAN GmbH - Otto-Hahn-Ring 6-18 - 64653 Lorsch - Tel.: +49 (0) 6251-86 69 80-0

M+W Dental

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Ⓒ **Type of application:** For the disinfection of all dental impression materials.

Notes on application: Dilute the concentrate with 10°C to 30°C warm water (at least drinking water quality) (use a dosing bottle). Place the respective impression in the prepared stock solution, the impression must be covered fully with the solution. Particularly with alginates and hydrocolloids, the exposure times must be observed (remove these immediately after completing the exposure time). After removal, rinse the impression with water. Do not mix with cleaning agents or other disinfectants! The stock solution can be disposed of in the waste water. Dosing quantities and exposure times should be adhered to as precisely as possible and should not be lower.

Exposure times:

Bacteria acc. to EN 13727, EN 14561, VAH, (high load):	2.0% 15 mins
Candida albicans acc. to EN 13624, EN 14562, VAH, (high load):	2.0% 15 mins
Enveloped viruses (e.g. HBV, HIV, HCV) according to DVV/RKI (high load):	2.0% 15 mins
TbB (M. terrae) acc. to EN 14348, EN 14563 (low load):	4.0% 15 mins 2.0% 60 mins

Safety notes: H302 Harmful if swallowed. **H314** Causes severe skin burns and eye

damage. **H410:** Very toxic to aquatic life with long-lasting effects. IF ON SKIN (or hair): Take off immediately all contaminated clothing. Rinse skin with water/shower. IF IN EYES: Rinse cautiously with water for several minutes. Remove contact lenses, if present and easy to do. Continue rinsing. Immediately call a POISON CENTRE/doctor. Disposal of contents / container in accordance with local / regional / national / international regulations. Contains N-(3-aminopropyl)-N-dodecylpropane-1,3-diamine, didecyldimethylammonium chloride.

Directive 648/2004/EC. Contains 5-15% non-ionic surfactants, <5% cationic surfactants, <5% EDTA salts, fragrances.

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Vertrieb durch Müller & Weygandt GmbH - Reichardsweide 40