ORANGE CLEANER

REF 000 250

Inhalt/Content/Contenu/
Zawartość/Obsah/Tartalma:
e 250 ml

Stand:080316

M+W Dental

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Ⓛ Cleaning solution on the basis of natural orange oils.

For cleaning of surfaces and instruments of soiling of all types, especially zinc oxide-eugenol pastes, silicon, alginates, adhesive lacquer, fats and remains of blood. The oily film can be washed off with water.

Before using on large surfaces with sensitive materials, for example, artificial leather, test first for material compatibility on a small hidden section. Do not use on acrylates and polycarbonates.

H 225 Highly flammable liquid and vapour. **H 315** Causes skin irritation. **H 319** Causes serious eye irritation. **H 317** May cause an allergic skin reaction. **H 336** May cause drowsiness or dizziness. **H 304** May be fatal if swallowed and enters airways. **H 410** Very toxic to aquatic life with long lasting effects. Keep away from heat/sparks/open

flames/hot surfaces. No smoking. Take precautionary measures against static discharge. Wear protective gloves and eye protection. **IF SWALLOWED:** Immediately call a POISON CENTER or doctor/physician. **IF IN EYES:** Rinse cautiously with water for several minutes. Remove contact lenses, if present and easy to do. Continue rinsing. Call a POISON CENTER or doctor/physician if you feel unwell. Do NOT induce vomiting. If skin irritation or rash occurs: Get medical advice/attention. **IF IN CONTACT WITH SKIN:** wash with copious amounts of water and soap. Store in a well-ventilated place. Keep container tightly closed. Dispose of contents/container in accordance with local/regional/national/international regulations. Contains orange terpene, 2-Propanol.

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Müller & Weygandt GmbH • Reichardsweide 40