Ketac™ Conditioner

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(03) ENGLISH

Product Description
 Ketac Conditioner is a polycrylic acid solution for removal of the smear layer after tooth preparation. Dentin conditioning with Ketac Conditioner significantly increases the bond of glass ionomer cement fillings.

Instructions for Use should not be discarded for the duration of product use.

Indications

- Dentin pretreatment prior to filling with glass ionomer cement

Precautionary Measures
 3M MDS/CS can be obtained at www.3m.com or from your local subsidiary.

Preparation

- ▶ After water spray, to remove any loose material already attached during preparation, and dry the cavity with air.

Pulp Protection
 Consider the possibility of pulp injury by applying tabs of a hard-setting calcium hydroxide material in order to protect the pulp from irritation.

Conditioning

- ▶ Dispense the conditioner from the bottle onto a pad and apply with a cotton applicator tip to the smear layer.
- ▶ Allow to react for 10 sec.
- ▶ Then rinse with copious amounts of water.
- ▶ Blow the cavity dry in only 2-3 short intervals with water and air-free air or dry air with cotton pellets. Do not over-dry!
- ▶ **The cavity should be just so dry that the surface has a mat shiny appearance.** Excessive drying can result in post-operative sensitivity after any delay.

Storage and Shelf-Life

Reseal bottle immediately after use.
 Store the product at 15-25 °C/59-77°F.
 Do not use after the expiration date.

Customer Information

No person is authorized to provide any information which deviates from the information provided in this instruction sheet.

Warranty

3M Deutschland GmbH warrants this product will be free from defects in material and manufacture. 3M Deutschland GmbH MAKES NO OTHER WARRANTIES INCLUDING ANY IMPLIED WARRANTY OF MERCHANTABILITY OR FITNESS FOR A PARTICULAR PURPOSE. User is responsible for determining the suitability of this product for their use. 3M Deutschland GmbH does not warrant the ability of the product for exclusive agents and 3M Deutschland GmbH's sole obligation shall be repair or replacement of the 3M Deutschland GmbH product.

Limitation of Liability

Except where prohibited by law, 3M Deutschland GmbH will not be liable for any loss or damage arising from this product, whether direct, indirect, special, incidental or consequential, regardless of the theory asserted, including warranty, contract, negligence or strict liability.
 Information as of December 2011

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3M, ESPE and Ketac are trademarks of 3M Company or 3M Deutschland GmbH, used under license in Canada.
© 2012, 3M. All rights reserved.