9. Use VivaStyle White daily without skipping a day. The varnish may be applied once or twice a day, depending on how the treatment fits into the daily schedule. A new dispensing cup and brush should be used for every application.

Notes

The following points should be noted in particular during the treatment:

- Avoid consuming citrus fruit and fruit juices, as they can promote hypersensitivity.
- Avoid the consumption of coffee, tea, red wine and colas as well as smoking to prevent the risk of staining the teeth.
- Only natural teeth are whitened. The appearance of existing restorations in the front teeth may no longer match that of the whitened teeth. Therefore, these restorations may have to be replaced.
- The treatment with VivaStyle White should be completed at least two weeks prior to the placement of tooth-coloured restorations.
- The teeth cannot be made to look whiter than their natural colour.
- The application of a thicker layer or several layers of varnish does not improve the results.
- The set varnish may appear to coat the teeth unevenly. This does not change the effectiveness of the product.

Warning

Avoid contact of the material with the skin, mucous membrane and eyes. After accidental contact, rinse the affected area with copious amounts of water. If contact with the eyes occurs, immediately rinse with plenty of water and seek medical advice.

Storage

- VivaStyle White is highly flammable
- Protect VivaStyle White from heat and sunlight
- Do not use after the date of expiration
- Store at < 30 °C / 86 °F.

Keep out of the reach of children!

The material has been developed solely for use in dentistry. Processing should be carried out strictly according to the Instructions for Use. Liability cannot be accepted for damages resulting from failure to observe the Instructions or the stipulated area of use. The user is responsible for testing the products for their suitability and use for any purpose not explicitly stated in the Instructions.

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Alcohol, Ethylcellulose, 0.1% Hydrogen Peroxide, Aqua, Panthenol, Aroma.

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Ivoclar Vivadent AG