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ENGLISH

1 - PRESENTATION

AIRnGO "CLASSIC" and **AIRnGO "PEARL"** polishing powders have been designed exclusively for supra-gingival prophylactic treatment. These powders can be used with all SATELEC® brand air polishers.

AIRnGO "CLASSIC" powders: Natural flavors available as an option: peppermint, lemon, raspberry, cola or neutral.
AIRnGO "PEARL" powder: Neutral

Packaging of AIRnGO powders:

- 20 g unit dose sachets for single use only - Boxes of 50 sachets.
- 250 g multi-dose bottle with spout.

2 - INDICATIONS

• Removal of dental plaque, (orthodontic treatment or before whitening), stain removal. Removal of temporary cement residues. • Polishing of teeth after scaling. • Inter-proximal cleaning, cleaning of fissures and grooves. • Cleaning of bands and brackets in the mouth. • Cleaning and preparation of the dental surface before etching.

3 - PROPERTIES

The abrasive properties of **AIRnGO** powders do not damage either the enamel or the gingiva.
AIRnGO "CLASSIC": Painless and without causing excessive bleeding, it is appropriate for supra-gingival maintenance treatments. Available in various flavors, it gives a sensation of freshness in the mouth.
AIRnGO "PEARL": No sodium bicarbonate. The powder consists of identical-sized microbeads. Makes it possible to perform an extremely active polishing that is remarkably effective and

painless for the patient. Suitable for patients on low sodium diets.

4 - COMPOSITION

AIRnGO "CLASSIC" powders: Sodium bicarbonate, hydrophobic silica, sodium saccharinate, excipients, natural essential oils or flavorings.
AIRnGO "PEARL" powder: Calcium carbonate.

5 - PRECAUTIONS FOR USE

Before using the powder, we suggest you establish the general state of health of your patient.
• Adhere to the conditions of use recommended by the air polisher manufacturer. • The patient and practitioner should wear appropriate protective accessories (goggles, masks). • Do not aim the jet directly onto the gingiva. • The patient should rinse the mouth using an antiseptic solution before and after treatment. • Apply Vaseline® to the lips before polishing. • Use a large aspirator cannula and hold it near the area to be treated (to reduce contamination by airborne bacteria). • In the case of accidental contact with the skin or eyes, rinse thoroughly with water. • Do not swallow.

6 - POSSIBLE SIDE EFFECTS

• Known allergy to one of the components. • Patients suffering from endocarditis. • Patients presenting an immune deficiency. • Patients on antibiotics, undergoing chemotherapy, radiotherapy, etc. • Patients on a low sodium diet. • Diabetics, asthmatics, or patients suffering from respiratory problems. • Hemophiliacs. • Pregnant women or breastfeeding mothers.

7 - INSTRUCTIONS FOR USE

Follow the instructions for use delivered with your air polisher. In general:
• Maintain a "jet-surface" angle of between 45° and 60°. • Maintain a "spray-surface" distance of about 6 mm.

- Make small circular movements on the tooth.
- Direct the jet towards the occlusal rim.

8 - STORAGE

Do not exceed the use-by-date shown on the external packaging. Close the bottle tightly after use (use contents quickly after opening). Discard the single-use dose sachet after use. Store at room temperature (< 25 °C) away from damp and light.

9 - DISPOSAL OF UNUSED POWDER

Do not discard in drains or rivers.
Do not discard with household waste.
Caution: Federal Law in the United States restricts the use of this equipment solely to qualified dental health professionals, or those under their control.

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