PROPHY TOP MINZE

REF 100 888

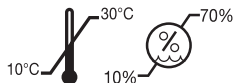
Inhalt/Content: e 300 g

- Ⓒ PROPHY TOP *MINT*
- Ⓕ PROPHY TOP *MENTHE*
- Ⓘ PROPHY TOP *MENTA*
- Ⓟ PROPHY TOP *MIĘTA*
- ⒸZ PROPHY TOP *MÁTA*
- Ⓗ PROPHY TOP *MENTA*

LOT

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Stand:030119

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Ⓢ PROPHY TOP MINT

Type of application

Prophylaxis powder based on sodium hydrogen carbonate for supragingival

removal of stubborn deposits and discolouration on tooth enamel. Suitable for all commercially available powder blasting devices.

Notes on application

Please observe the instructions for use and the maximum filling level of the device manufacturers. Not suitable for the removal of deposits or concretions in gingival pockets, also do not direct the powder jet onto the gingiva or other soft tissue. Do not expose dentine and enamel caries to jet blasting as higher abrasive values are to be expected here. Composite, compomer, amalgam and gold fillings must also not be jet blasted, as this can lead to roughening. Patients suffering from chronic bronchitis or asthma must never be treated with a powder jet device. Patients on a low sodium diet (e.g. high blood pressure) should not be treated with the powder as it contains sodium hydrogen carbonate. Patients suspected of being allergic to flavouring substances must not be treated with the powder. Patient and operator must wear eye protection. The operator must wear a mask and gloves. Use a high performance suction device. Apply petroleum jelly to the patient's lips. For dental use only. Keep container closed. **Shake thoroughly before use!**

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