# help:ex plaque p

Plaque Remover + Neutralizer | REF. 6700 0000

## DEUTSCH

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## ENGLISH

**Product description:**

help:ex plaque p is an acid-based cleaning agent in a powder form. It is used for cleaning dental restorations e.g. full dentures, splints and orthodontic appliances. The cleaning process can be carried out in the SYMPRO denture cleaning unit and in an ultrasonic unit. A neutralizer in the form of a powder is available for the easy and safe disposal of the cleaning agent after use.

**Indication:**

A cleaning agent for the removal of plaque and tartar deposits on dental restorations.

Only to be used by trained dental personnel.

**Contraindication:**

Application in the mouth or on the body is not permitted.

The use of the cleaning agent on restorations belonging to patients with known allergies to individual constituents is contraindicated. The cleaning agent is not intended for multiple use.

Restorations containing feldspar and leucite ceramic must not be cleaned with the product must not be cleaned with the product (risk: pitting). It is not permitted to store the ready-to-use cleaning solution for later use. It is not intended for use in the SYMPRO mini cup. The mixed cleaning agent should not be disposed of without prior neutralization. The neutralizer is only compatible with the help:ex plaque p cleaning powder.

**Special hazard warnings:**

Please read the cleaning agent safety data sheet before use. Download under [www.renfert.com](http://www.renfert.com)

Please wear personal protective gear when handling this product. (protective gloves, eye protection).

Avoid skin contact. The constituents can cause skin irritations after contact with the skin and severe eye irritation after contact with the eyes.

**After inhalation:** Provide fresh air (there may be a dust cloud when pouring in).

**After skin contact:** After skin contact, wash hands immediately with plenty of water.

**Skin irritation:** Seek medical help.

**After eye contact:** Rinse the eye thoroughly for several minutes. If contact lenses are worn, remove if possible and continue to rinse with water. Seek medical help.

**After swallowing:** Rinse mouth immediately and drink plenty of water afterwards.

DO NOT induce vomiting. Call doctor immediately.

In case of doubt or if symptoms are present, seek medical advice.

Please observe the dosage recommendations on the sachets in order to achieve the required cleaning effect (1. Cleaning powder, 2. Cleaning procedure, 3. Neutralization powder)!

Observe the operating instructions of the cleaning units to avoid damage to the equipment and reduced cleaning performance.

It is essential to rinse the cleaned object thoroughly to avoid any cleaning agent residues from entering the patient's mouth.

Check the dental restoration for possible damage before inserting into the patient's mouth.

Observe the national hygiene regulations in order to avoid any cross contamination. To protect working surfaces, the use of an alkali-resistant surface covering is recommended. Remove cleaning fluid residues from working surfaces immediately with a damp cloth and rub dry with a dry cloth.

**Application:**

Use only one sachet per application. (An overdose will not improve the cleaning performance).

help:ex plaque p is designed for single-use only, in order to prevent cross contamination and reduced cleaning performance. Before applying, any loose contaminants must be rinsed from the restoration under running water. Prior to the cleaning procedure, observe the disinfection specification according to national hygiene regulations.

**Use in the SYMPRO denture cleaning unit:  
Before use, it is essential to read the Instructions  
for Use of the SYMPRO denture cleaning unit!**

- ▶ Open the SYMPRO cleaning bowl.
- ▶ Ensure that the cleaning needles (approx. 75g) are inside the cleaning bowl.
- ▶ Fill the cleaning bowl up to the "min. marking" with water.
- ▶ Place the restoration to be cleaned into the cleaning bowl.
- ▶ Open one sachet of cleaning powder and carefully empty the bag into the bowl (try to avoid dust clouds).
- ▶ Close the bowl (ensure that the lid / seal is correctly in place) and insert it into the SYMPRO unit.
- ▶ Start the cleaning procedure (approx. 10–30 min at level I–III or 1200–2000 rpm. See information on the speed and running times in the Instructions for Use for the SYMPRO unit).
- ▶ Then, the restoration must be thoroughly and intensively (min. 6 seconds) rinsed under running water.
- ▶ If necessary, the procedure can be repeated.
- ▶ Before inserting into the patient's mouth, the restoration must be checked for possible damage and treated in accordance with national hygiene regulations.
- ▶ After the cleaning process has finished, open a bag of neutralization powder and pour it into the bowl containing the used cleaning solution.
- ▶ Close the lid and place it into the SYMPRO unit.
- ▶ Start the neutralization procedure with a running time of 1 minute (level III or 2000 rpm).
- ▶ Open the lid and hold the cleaning needles by placing a magnet underneath the SYMPRO bowl.
- ▶ Dispose of in compliance with national regulations.

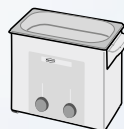
**Note:**

A color change of the liquid after neutralization is part of the chemical process. The instructions for hygienic reprocessing and storage of the system components (e.g. the cleaning bowl) can be found in the Instructions for Use of the SYMPRO denture cleaning unit.

## ENGLISH

**Use in the Easyclean ultrasonic unit:**  
**Before use, it is essential to read the Instructions for Use of the Easyclean ultrasonic unit!**

- ▶ Fill a glass beaker with 200 ml water.
- ▶ Open one sachet of cleaning powder and carefully empty the bag into the beaker (try to avoid dust clouds).
- ▶ Using a suitable instrument, stir the powder for min. 30 seconds (a residue is left at the bottom).
- ▶ Place the restoration to be cleaned into the glass beaker, and place the beaker in the ultrasonic unit (observe the use of the required support).
- ▶ **Note:** In order to avoid damage to the device, do not place any item directly into the tank.
- ▶ Start the cleaning process (temperature max. 45 °C)
- ▶ **Note:** The cleaning duration depends on the level of contamination on the restoration and is approx. 30 minutes.
- ▶ After the process has finished, remove the restoration from the glass beaker.
- ▶ Then, the restoration must be thoroughly and intensively (min. 6 seconds) rinsed under running water.
- ▶ If necessary, the procedure can be repeated.
- ▶ Before inserting into the patient's mouth, the restoration must be checked for possible damage and treated in accordance with national hygiene regulations.
- ▶ Dispose of in compliance with national regulations.

**Note:**

When using the ultrasonic unit, neutralization is not possible! Please observe the Instructions for Use for information on cleaning the unit equipment.

**Disposal:**

Within Germany, neutral waste water can be disposed of in the drainage system (WHG – §55). Non-neutralized, used cleaning liquid must be disposed of in the hazardous waste disposal system.

Unused neutralizer can be disposed of via the municipal waste system (non-hazardous waste). Please observe the national regulations for waste disposal.

**Storage:**

Store in a dry environment and protect against direct sunlight. Storage temperature from 5 °C to 40 °C. Already opened sachets must not be stored.

**Shelf life:**

For the shelf life, please see the product label. Do not use once this date has expired

**Constituents:**

Cleaner (powder): Sulphamic acid, anionic surfactants <5%, fillers

Neutralizer (powder): Mixture of mineral salts, fillers

**Disclaimer:**

This product has been developed and is intended exclusively for professional use by trained dental professionals. It must be handled in accordance with the information provided in this instruction document. The manufacturer assumes no liability for damages that arise from improper handling or processing.

In case of complaints and events in connection with the medical device, please inform Renfert immediately. Renfert will investigate the matter and report serious incidents to the authorities in the concerned member states.

	Unique Device Identifier		UK conformity
	Professional use		Storage
	Please observe the instructions for use*		Shelf life
	EC conformity		Use protective eyewear
	Manufacturer specifications		Wear protective gloves
	Not intended for reuse		Not for intraoral use
	Batch designation		Warning
	Attention: see accompanying document*		Medical device

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Renfert GmbH