These instructions cover all Nupro® Prophylaxis Paste Flavors/Grits.

Nupro® is a premium Prophylaxis Paste, available in fluoride-containing and fluoride-free formulations, which gently removes dental plaque and staining.

Nupro® Prophylaxis Paste is available in the following grits:

Fine (Gold)

- Ideal for younger patients and prophyl procedures requiring minimal tooth abrasion.

Medium (Blue)

- Suitable for most cleaning procedures where a high level of polish is desired.

Coarse (Red)

- For patients with substantial stain build-up due to smoking or excessive use of stain producing beverages.

Plus (Green)

- An extra heavy duty cleaner for stain build-up difficult to remove with conventional prophyl paste.

INDICATIONS FOR USE

To be used for cleaning and polishing procedures as part of a professionally administered prophylaxis treatment.

INGREDIENTS

Medicinal Ingredients: 2.72% Sodium Fluoride (1.23% Fluoride Ion) (for fluoride containing pastes only)

Non-Medicinal Ingredients: Pumice, Glycerin, Water, Diatomite (fine grit only), Sodium Silicate, Flavor, Methyl Salicylate, Monosodium Phosphate, Sodium Carboxymethylcellulose, Sodium Saccharin, FD&C Yellow #6 (fine grit), FD&C Blue #1 Lake (medium grit), FD&C Red #40 Lake (coarse grit) and D&C Yellow #10 and D&C Green #5 (plus grit)

CONTRAINDICATIONS

Nupro® Prophylaxis Paste with and without fluoride is contraindicated in patients with a known allergy to any of the components.

WARNINGS

- Avoid prolonged contact with inflamed tissue.
- Avoid eye contact. If contact with eyes occur, immediately flush eyes with copious quantities of water for 15 minutes. Seek medical attention if irritation or discomfort persists.
- Avoid using fluoride-containing pastes in individuals with hypersensitivities to fluoride as adverse reactions can occur. In the case of adverse reactions, seek medical attention immediately.
- Unit dose cups contain enough prophyl paste for a single procedure. Each cup is intended to be used once for a single patient. Reuse of an unfinished cup for another patient could result in cross-contamination.
- Reuse of contaminated instruments to remove paste from multi-use jars can result in cross-contamination.

PRECAUTIONS

- Prophylaxis pastes containing fluoride are not considered a replacement for other forms of fluoride therapy.
- Nupro® Prophylaxis Pastes are to be used only by individuals professionally trained to perform a dental prophylaxis.
- During use, wear protective glasses and gloves.
- Prior to pit and fissure sealing, Nupro® Prophylaxis Paste without fluoride should be used.

ADVERSE REACTIONS

For fluoride containing pastes, the following adverse reactions are possible in individuals hypersensitive to fluoride: salivation, gastric distress, headache, weakness, tremors, spasms, convulsions, eczema, atopic dermatitis or urticaria.

RESIDUAL RISKS

Using the paste on a patient with inflamed tissue could cause further irritation.

Failing to wear protective glasses can result in eye irritation on contact with the paste.

Applying fluoride-containing pastes to individuals with fluoride hypersensitivities can result in serious illness.

Reactions may occur in hypersensitive individuals or in individuals with allergies to any of the raw materials of the paste.

Use on restorations may affect the integrity of the restorative material and could lead to caries and/or periodontal disease.

Cross-contamination can lead to patient infection or illness.

STEP-BY-STEP INSTRUCTIONS

When using bulk jars, sufficient paste for one procedure should be removed and placed in a clean dappen dish or other suitable container. Close lid immediately after removing paste. When using single-use cups, insert the cup into a Nupro Prophyl Paste Holder for easier handling.

Using a disposable prophyl angle such as Nupro® Revolv® or Nupro Freedom® Disposable Prophyl Angle or a conventional metal prophyl angle, fill the cup by pressing the face of the cup into the paste while running the angle at low speed.

Throughout the procedure, sufficient paste should be maintained on the cup to avoid overheating of the tooth surface. Saliva evacuation is recommended to avoid excessive dilution and swallowing of the paste. Rinse full mouth with water spray.

STORAGE

When using the bulk jars, close lid immediately after use to avoid moisture loss. Do not store above 25°C (77°F). Keep out of direct sunlight.

EXPIRATION DATE AND LOT NUMBER The ReOrder Number, Lot Number, and Expiration Date should be referenced in all correspondence requiring identification of the product. Do not use after expiration date.

SAFETY DATA SHEET is available on our website, www.dentsplysirona.com or by contacting Customer Service at 1-800-989-8826.

SUMMARY OF SAFETY AND CLINICAL PERFORMANCE

All SSCP documentation can be found on the Eudamed database.

All SSCP documentation can be found on the Eudamed database.

Any serious incident that has occurred in relation to this device should be reported to the manufacturer and the Competent Authority of the Member State.

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Dentsply DeTrey GmbH