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Hämostatikum Al-Cu

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CE 0482

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Aluminiumchlorid, Kupfer-II-chlorid, Aqua dest.

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20 ml (0.7 fl oz) liquid

Ingredients:

Aluminium chloride, copper(II) chloride, distilled water

Indication / purpose:

For stopping smaller bleedings of the oral mucosa and the pulp.
Side effect: germ reduction.

Application:

Slightly press a cotton pellet with a drop of Hämostatikum Al-Cu onto the bleeding area. Take care that there is neither saliva nor mucous secret between the bleeding area and Hämostatikum Al-Cu. Otherwise the effect would be reduced importantly. In case of bleeding in the root canal, Hämostatikum Al-Cu could be applied also with slight pressure, by help of a cotton thread or paper edge. Carefully remove mucous secret of infected areas in the oral cavity in the mouth before applying Hämostatikum Al-Cu.

Characteristics:

An aluminium salt with small anions acts as coagulant for ensuring an optimum diffusion. Copper chloride acts astringent and effects in addition a germ reduction. Due to the large anions it is lightly liposoluble.

Attention! This medical device could be used in pregnancy, lactation and for children older than 6 years. Due to inadequate experiences for using this medical device in pregnancy and lactation, this medical device should only be used under medical advice.

Contraindications and combined effects: none.

Warning notice:

Do not bring in contact with eyes!

Close bottle tight!

Use before: see packing imprint.

Product shall be sold only to dental professionals.

All serious incidents associated with the product should be reported to the manufacturer and the competent authority of the country in which the user is established.

Do not use the preparation after the expiry date. Care for an appropriate waste disposal. Empty the bottles completely before entering them into a recycling system. Unneeded rests are retracted free of costs by the producer.

If you have any questions or problems concerning this medical device please contact HUMANCHEMIE GmbH directly, we will willingly give you advice.

HUMANCHEMIE GmbH, Hinter dem Krüge 5,
DE-31061 Alfeld (Leine), Lower saxony, Germany

Tel. +49 (0) 51 81 - 2 46 33
Fax +49 (0) 51 81 - 8 12 26
www.humanchemie.de