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GB Processing instructions

1 Intended purpose

Denture base materials are intended for the fabrication of splints, partial dentures, full dentures and model dentures.

2 Description of product and users

2.1 Product description

Denta-Fast NF is a cold-curing methylmethacrylate-based denture acrylic for repair, extension or relining of existing restorations using the flow technique. The powder is available in various colours, in some cases with veins.

2.2 Users

For use by laboratory technicians in a dental laboratory

3 Composition

Powder: Bead polymer of polymethyl methacrylate (PMMA) Pigments, initiators

Liquid: Methyl methacrylate, tetramethylene dimethacrylate, initiators, stabilisers

4 Indications

Repairs, extensions and relinings of existing restorations using the flow technique

5 Contraindications

Do not use in the case of a known allergy to one of the components.

6 Warnings

Liquid: Contains methyl methacrylate, tetramethylene dimethacrylate.

Danger. Highly flammable liquid and vapour. Causes skin irritation. May cause an allergic skin reaction. May cause respiratory irritation.

Powder: Contains dibenzoyl peroxide, methyl methacrylate. May cause an allergic reaction. Harmful to aquatic life with long lasting effects.

7 Safety instructions

Liquid: Keep away from heat, hot surfaces, sparks, open flames and other ignition sources. No smoking. Use explosion-proof [electrical/ventilating/lighting] equipment. Avoid breathing vapours/spray. IF ON SKIN (or hair): Take off immediately all contaminated clothing. Rinse skin with water [or shower]. Store locked up

Powder: Avoid release to the environment.

8 Interactions with other medicinal products

The dentist should consider known interactions and cross-reactions of the medical device with other materials already in the patient's mouth before using the product.

9 Application / Preparation

9.1 Preparatory work

Produce the plaster cast and wax-up of the teeth as usual.

The fabricated tooth set in wax is secured with plaster or silicone (matrix/flask). Then the wax is eliminated as usual.

To prevent artificial teeth from breaking off, they must be roughened, have undercuts made, and be wetted with monomer or a suitable bonder.

The surfaces of the plaster models are separated with an alginate separating agent according to manufacturer's instructions.

9.2 Mixing ratio

- 5 parts by weight powder, 3 parts by weight liquid
- Can be measured out freely
- Veins can be included, if desired.
- Please observe separate manufacturer's instructions.

9.3 Repairs, extensions and relinings of existing restorations using the flow technique

Mix powder and liquid (according to the mixing ratio) and stir with a spatula for 15 seconds. When freely mixing, place Denta-Fast NF liquid in the mixing vessel and sprinkle powder until a low level of liquid remains.

Mix for 15 seconds.

The acrylic dough then has a runny consistency, forms drops, and can be processed immediately. Once the surface of the denture has been roughened, the runny dough should be dripped onto it and spread evenly.

After the dough has attained a plastic consistency, it must not be touched or contoured.

Larger amounts of Denta-Fast NF should be applied in layers to prevent porosity. The only method of ensuring reliable, nonporous setting is to cure the material under pressure.

The processing window is 5 minutes.

The polymerisation time in a pressure polymerisation device is 15 min., at a pressure of 2- 5 bar and a water temperature of 45°C.

9.4 Finishing the material

After polymerisation lift the workpiece off the model and finish with suitable tools.

Silicone polishers (square-edge or knife-edge wheels or cylinders), tungsten carbide burs and diamond instruments are suitable for finishing.

Sandpaper or pumice stone is used for pre-polishing.

9.5 Polishing

The material is polished with goat bristle brushes, polishing paste and soft cotton wool discs. Careful surface finishing and polishing is necessary for an optimal result.

10 Notes on processing

To further reduce the risk of intolerance reactions, we recommend placing the denture in lukewarm water for at least 12 hours prior to insertion.

11 Troubleshooting / FAQ List*

12 Storage and handling information

Storage temperature 10-25°C / 50-77°F. Close container carefully.

13 Shelf life

The maximum shelf life is printed on the label of each pack. Do not use after the expiry date.

14 Warnings on side effects

With proper preparation and use of this medical device, adverse effects are extremely rare. However, immune reactions (such as allergies) or local discomfort cannot in principle be ruled out completely. All serious incidents which occur in connection with the use of this product are to be reported to the manufacturer indicated below and the competent authority in each case.

15 Instructions for disposal:

Leftover quantities and packaging materials are to be disposed of according to the local and/or statutory regulations.

* 11 Troubleshooting

Fault	Cause	Corrective action
Acrylic does not cure properly	Incorrect mixing ratio	– Refer to the instructions for use – Ensure correct mixing ratio
Acrylic is whitish at base	– Inadequate insulation – Cast not steeped in water	– Ensure adequate insulation – Check insulating materials if necessary – Allow cast to steep for approx. 10 min in lukewarm water
Acrylic forms bubbles at base	– Cast not steeped in water	Allow cast to steep for approx. 10 min in lukewarm water
Acrylic surface white/porous	Acrylic injected too late or placed in pressure vessel too late	Observe the processing times
Differences in colouration following repairs	Incorrect mixing ratio	Ensure correct mixing ratio
Acrylic breaks when lifted out	Inadequate insulation	Check insulation
Acrylic/denture has whitish streaks	– Inadequate mixing – Too much powder used	– Carefully stir acrylic dough with spatula – Ensure correct mixing ratio
Acrylic sets too fast	Excessively high temperatures/ excessively long work steps	Observe the temperatures/ processing times
Splint breaks	Contouring too thin/unsuitable	Ensure sufficient layer thickness (at least 2-3 mm)

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M+W Dental Müller & Weygandt GmbH · Reichardsweide 40 · 63654 Büdingen/Germany · Tel.: +49 (0) 60 42 - 88 00 88
MANI MEDICAL GERMANY GmbH · Hertha-Sponer-Straße 2 · 61191 Rosbach v.d. Höhe/Germany