3M ESPE

Ketac® Bond

Glasionomer Base Material – Radiopaque

Röntgensichtbar

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Dental Products
Carl-Schurz-Str. 1
41453 Neuss - Germany

3M ESPE Company Carlsburg/MSD Informa

USA: 1-800-634-2249 and Canada: 1-888-363-3085,

3M, ESPE and Ketac are trademarks of 3M

© 2016, 3M, All rights reserved.

ENGLISH

Product Description

Ketac® Bond is a fast-setting glass ionomer cement for manual mixing. The feature of the material to chemical bond to enamel and dentin provides for gentle preparation. Ketac Bond releases fluoride ions and is radiopaque. Ketac Bond is available in one color, yellow.

* For details on Ketac® Conditioner please refer to the pertinent Instructions for Use.

Instructions for Use should not be discarded for the duration of product use.

Indications

• Bonded linings under composite and amalgam

• Extended fissure sealing without acid etching

• Zinc fill-ups

Precautionary Measures

3M MSDS can be obtained at www.3m.com or from your local subsidiary.

Preparation

• Only remove the caries substance, no undercuts required.

Protection of the pulp

• Cover areas in close proximity to the pulp by applying spots of a hard-setting calcium hydroxide material in order to protect the pulp from irritation.

Conditioning

• After chemical conditioning through to the tooth the smear layer produced in the preparatory steps must be carefully removed by applying Ketac Conditioner to the prepared surfaces and allowing the substance to react for 10 to 30 sec.

• Use the brush with copious amounts of water.

• Blow the cavity dry in only 2-3 short intervals with water and oil-free air or dry air with cotton pellets. Do not over-dry!

• **The cavity should be just so dry that the surface has a matt shiny appearance.** Excessive drying can result in post-operative sensitivity after filling.

• Avoid further contamination.

Dosing

For dosing, please adhere to the following guidelines:

• Before the first use, screw the drop dosing device onto the storage bottle.

• Shake bottle vertically for dosing.

• The drop dosing device must be free from any dried residue of the filling.

• Shake the bottle before you use to render the powder free-flowing.

• Place the powder and liquid doses in adjacent spots on the mixing spat.

• Carefully invert both bottles directly after use!

For base linings it is useful to have an indirect condensation consistency.

Use a mixing ratio (weight ratio) of 4:2 parts of powder : 1 part of liquid (1 drop).

The enclosed dosing spoon gives an approximate amount of powder to add.

Use the mixing ratio to add the liquid in small portions until the consistency is adequate.

For exterior fissure sealings, a creamy consistency is suitable. Use a mixing ratio (weight ratio) of 3.5 parts of powder (approx. one level dosing spoon) : 1 part of liquid (1 drop). For powder dosing, place the free-flowing powder on the enclosed dosing spoon and scrape the spoon level on the plastic insert of the bottle.

Mixing

• Process Ketac Bond at room temperature (20-25°C/68-77°F). Use a metal or plastic dosing spatula to mix powder and liquid on a mixing pad or glass slab. Spread out the paste repeatedly until the consistency is homogeneous.

Application

Ensure that there is no contamination from water, saliva, etc., entering the working area for the entire duration of application; the working area must be kept dry.

Aesthetic fillings can be processed with the so-called sandwich technique, in which the glass ionomer cement base filling replacing the dentin is covered by a layer of composite material according to the following procedure:

• Apply the desired quantity of Ketac Bond with a suitable instrument leaving the margins of the dentin exposed.

• After Ketac Bond has set, etch both enamel and dentin and then apply the desired composite using a suitable bonding material.

• Process this material in accordance with manufacturer's recommendations.

• Lining located below amalgam does not have to be etched and bonded.

• For fissure sealings and core build-ups, fill the cavity using a suitable instrument. Wait a minimum of 4 minutes from start of mixing prior to preparation of the harness: Ketac Bond with rotary instruments.

• Ketac Bond adheres to metal instruments. Therefore, residual quantities of the cement should be washed off with cold water before they set on the instrument.

Times

At an ambient temperature of 23°C/74°F and 50% relative humidity the following times apply:

Mixing min/sec 00:30

Processing including mixing 02:00

Setting from start of mixing 04:00

Setting from the mouth 02:00

The following factors affect the setting time:

• At higher temperatures than indicated above the time available for processing is shortened, whereas it is prolonged at lower temperatures.

• Mixing on a chilled glass slab also affects the processing time.

• Adding more powder than would be equivalent to the stated mixing ratio shortens the time available for processing.

Storage and Shelf-life

Store powder in its original bag and protect from moisture.

Store your product at 15-25°C/59-77°F.

Do not use after the expiration date.

Customer Information
No person is authorized to provide any information which deviates from the information provided in this instruction sheet.

Warranty

3M Deutschland GmbH warrants this product will be free from defects in material and manufacture. 3M Deutschland GmbH MAKES NO OTHER WARRANTIES INCLUDING ANY IMPLIED WARRANTY OF MERCHANTABILITY OR FITNESS FOR A PARTICULAR PURPOSE. User is responsible for determining the suitability of the product for user's application. If this product is defective within the warranty period, your exclusive remedy and 3M Deutschland GmbH's sole obligation shall be repair or replacement of the 3M Deutschland GmbH product.

Limitation of Liability

Except where prohibited by law, 3M Deutschland GmbH will not be liable for any loss or damage arising from this product, whether direct, indirect, special, incidental or consequential, regardless of the theory asserted, including warranty, contract, negligence or strict liability.

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