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Avril 2019

HEMOSTASYL KIT

1 - PRESENTATION: Kit containing: 2 x 2g syringes of HEMOSTASYL Paste (about 30 uses) and 40 HEMOSTASYL single-use angled application cannulas. Refills: HEMOSTASYL Paste: box with 2 x 2q syringes of paste - HEMOSTASYL Cannulas: box with 40 single-use angled application cannulas.

2 - PROPERTIES/3 - COMPOSITION: see HEMOSTASYL Paste IFU. **4 - INDICATION**: HEMOSTASYL Paste is designed to be used in all cases of moderate bleeding occurring in routine dental practice (scaling, etc.).

5 - PRECAUTIONS FOR USE: Cannulas are disposable (risk of inter-patient contamination). DISPOSE OF used cannulas after use. See HEMOSTASYL Paste IFU. 6 - CONTRA-INDICATIONS/7 - INSTRUCTIONS FOR USE: see HEMOSTASYL Paste

8 - STORAGE: See HEMOSTASYL Paste IFU. After use, imperatively remove and discard the used cannula. Do not store cannulas in contact with syringes. Store new cannulas in their original packaging, away from syringes (risk of corrosion). 9 - DISPOSAL: Recycle or dispose of according to current legislations, rather by a waste collector or an approved company. Do not dispose of in the environment. Professional dental products.

HEMOSTASYL

1 - PRESENTATION: 2 x 2g syringes of HEMOSTASYL Paste (about 30 uses). **2 - PROPERTIES**: HEMOSTASYL is an original paste to be used as a haemostatic dressing with a mechanical action. The main characteristics of HEMOSTASYL Paste are as follows: • thixotropic properties • good adhesion to the gingival mucosa, without compression • presence of aluminium chloride which reinforces its mechanical haemostatic effect • placed directly in the mouth with the single-use angled cannula • easily located due to its colour • easily removed by gentle water spray without resumption of bleeding • painless method when HEMOSTASYL Paste is used on a healthy periodontium • contact time: 2 minutes max. 3 - COMPOSITION: Aluminium chloride. Excipients.

4 - INDICATION: HEMOSTASYL Paste is designed to be used in all cases of moderate bleeding occurring in routine dental practice (scaling, etc.). **5 - TARGET POPULATION: •** This product is intended to patients (adults and children) not subject to the contraindications and/or to the precautions for use. • None of the compounds in the formula present any danger when the product is used in pregnant or breastfeeding women or in children. In addition, the product is for occasional, highly localised use, the quantities administered are small (of the order of 0.15g per application i.e. 0.0225g of aluminium chloride) and the contact time is limited (2 minutes maximum). There is therefore very little transfer **6 - PRECAUTIONS FOR USE**: HEMOSTASYL Paste is not indicated in the case of heavy bleeding: in this case, use ETIK™ COLLAGENE for example. Ensure complete removal of HEMOSTASYL Paste. Chemical residues of haemostatic agents may reduce the adhesive strength of bonding agents to dentine: before bonding, rinse with water for at least 10 seconds. In case of contact with the eyes, rinse with plenty of water. Do not swallow. Keep out of the reach of children. Respect the recommended contact time (risk of pain and/or irritation if contact > to

7 - CONTRA-INDICATIONS: Extraction sockets. Root canal surgery. Known allergy to aluminium chloride. Do not use if the aspect of the paste has changed (hardening, phase separation). Do not use in a site where the product may not be completely removed, such as periodontal pockets. Under normal conditions of use and if the precautions for use described in the instructions are followed, there are therefore no contraindications for using HEMOSTASYL Paste in pregnant or breastfeeding women or in children.

8 - INSTRUCTIONS FOR USE: • Preparation: Before each use, prepare the

syringe by taking a small amount of product out of the patient's mouth. Open the syringe. Screw the single-use angled application cannula onto the tip of the syringe. • Application: The paste is applied directly onto the bleeding gum to completely cover the bleeding site, without exerting any pressure on the gum with the cannula. Leave in place for 2 minutes max. • Removal: Remove HEMOSTASYL Paste by gentle water spray and simultaneous aspiration. Verify that HEMOSTASYL Paste is completely removed from the qum. • <u>Disinfecting</u> the outside of the syringes: Once the single-use cannula has been removed and discarded, the outside of the syringe must be cleaned and decontaminated using (aldehyde-free) disinfectant wipes in accordance with the manufacturer's directions for use.

9 - STATEMENT OF SERIOUS SIDE EFFECTS: Any serious incident involving the device must be notified to the manufacturer and to the competent authority of the Member State where the use and/or the patient is established. **10 - STORAGE**: Tightly close the syringe. Shelf-life: 24 months from the

manufacturing date. Do not use past the expiry date indicated on the outer packaging. Store at a temperature between + 5°C/+ 41°F and + 25°C/+ 77°F. 11 - DISPOSAL: Recycle or dispose of according to current legislations, rather by a waste collector or an approved company. Do not dispose of in the environment. Professional dental product. April 2019

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