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Produktname

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M+W Paste Carriers - Dental Root-Canal Instruments

Intended use

It is used for Root canal filling or drug delivery.

Structure and composition

It mainly consist of operative part and shank.

Main materials

Operative part: Stainless Steel

Shank: Copper alloy

Contraindications

The product contains Nickel and Chromium and should not be used on patients with known allergic sensitivity to these metals.

Warning

- The product is used only by formally well-trained professional dental practitioners.
- This product is limited to dental treatment purposes and is prohibited for operations other than dental treatment.
- Follow good dentist practice, especially using the rubber dam system during pulp treatment.

Attention

- For your own safety, please wear personal protective equipment (gloves, glasses, mask).
- Check whether the product is within the prescribed validity period before use.
- This product is single use, prohibited for two times.
- The Paste Carrier is used together with the root canal motor, with an allowable rotational speed range of 250-800 rpm. Use the indicated rotation method at the recommended speed of the label.
- In anatomical complexity, the maximum torque can differ from the manufacturer's recommended value.
- The clumping chuck of the root canal motor used with the paste carrier shall befit into the shank of the type I specified in ISO 1797.
- Follow the recommended usage steps for use.
- Keep the UDI information on the label.
- Production year: please refer to the label.
- Do not use the instruments when storage conditions

are suspected

- If you have any questions about the product identity, do not use it.
- Check the status of the device before using root canal operation. The device shall not be used if it is damaged or shows signs of wear.
- Serious accidents related with this product shall be reported to the manufacturer and national authorities.

Use steps

Attach the paste to the surface of the paste carrier, rotate the paste carrier to deliver the paste into the root canal, and repeat the operations until the entire root canal is filled with the paste.

Cleaning disinfection/sterilization

Materials needed

- Gloves, masks, blouse as recommended by the manufacturer of the cleaner and detergent
- Running or deionized water
- Detergent
- Container
- Ultrasonic bath or washer-disinfector
- Class B sterilisation apparatus

Product storage conditions

Take it lightly and put it lightly during transportation and storage, do not stack it, to prevent damage to the packaging materials. Avoid storage in high temperature, high humidity and direct sunlight, and the products should be stored at room temperature.

Waste treatment

After use, follow good practice for dental root-canal instruments and must be placed in a solid container that should be used for collecting cutting or bonding instruments (e. g. needle or disposable scalpel).

Term of validity

The product is valid for 5 years

Manual operation	1	Cleaning Insert the products in the ultrasonic bath. Process for 10 to 30 minutes. <i>Remarks:</i> Follow instructions and observe water quality, concentrations and cleaning time stated by the manufacturer of the cleaning solution.
	2	Rinsing Rinse the products with tap water at 20-40 °C for 1min. <i>Remark:</i> It is recommended to use deionized water
	3	Drying Dry the products with compressed air until product is visibly dried
Automatic operation	1	Cleaning/Rinsing/Drying Place the instruments in the tray of the slide-in trolley of the washer-disinfector Preparation and drying.
	2	
3		
4	Packaging Place the instruments in a paper-plastic pouch for steam sterilisation compliant with ISO11607 and EN 868. <i>Remarks:</i> <ul style="list-style-type: none"> • For sharp devices that are not contained within a box silicone tubes should be placed around the devices to prevent piercing of the packaging. • Seal the pouches according to the recommendation of the pouch manufacturer. If a thermo-sealer is used, the process must be validated and the thermo-sealer must be calibrated and qualified. 	
5	Sterilisation Sterilise the products with steam vapour Apparatus: class B Recommended sterilization parameters: The setting temperature is 134 °C and the duration is 18 min. <i>Remarks:</i> <ul style="list-style-type: none"> • When sterilising multiple instruments in one autoclave cycle, ensure that the steriliser's maximum load is not exceeded. • Place the pouches in the steam steriliser according to the recommendation given by the steriliser manufacturer. • Use only pre-vacuum air removal steam sterilizers that meet the requirements of EN 13060(class B, small steriliser) and EN 285 (full size steriliser), with saturated steam. 	
6	Storage Store the products in a dry, clean and dust-free environment at temperatures specified for paper-plastic pouches by the steam sterilisation manufacturer. <i>Remark:</i> Check the packaging and the medical devices before use(packaging integrity, no humidity and use-by date. In case of damage, a complete rework should be performed.	

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