Program structure

Program	Name	Description	Predrying	Program duration ^[4]
1	IPS e.max CAD Superspeed crystallization HT, MT, LT / Spray	Program for the superfast crystallization of a maximum of two small restorations made from IPS e.max CAD HT, MT or LT. No abutments possible. Optional IPS e.max CAD Crystall./ Spray glaze possible.	Crystallization	11:10
2	IPS e.max CAD Speed crystallization HT, MT, LT / Glaze	Program for the fast crystallization of a maximum of three single-tooth restorations made from IPS e.max CAD HT, MT or LT. No abutments possible. Optional IPS e.max CAD Crystall./ Spray or Paste glaze and Shades/Stains possible.	Glaze	16:50
3	IPS e.max CAD Crystallization HT, MT, LT / Glaze	Program for the crystallization of a maximum of four single-tooth restorations or one 3-unit bridge made from IPS e.max CAD HT, MT or LT. Optional IPS e.max CAD Crystall./ Spray or Paste glaze and Shades/Stains possible. Add-on possible.	Glaze	24:40
4	IPS e.max CAD Crystallization HT, MT, LT, MO, Impulse / Glaze	Program for the crystallization of a maximum of four single-tooth restorations or one 3-unit bridge made from IPS e.max CAD HT, MT, LT, MO or Impulse. Optional IPS e.max CAD Crystall./ Spray or Paste glaze and Shades/Stains possible. Add-on possible.	Glaze	29:30
5	IPS e.max CAD Corrective/Stain/Glaze firing with IPS e.max CAD Crystall./	Program for the Corrective/Stain/Glaze firing of a maximum of four single-tooth restorations or one 3-unit bridge made from IPS e.max CAD with IPS e.max CAD Crystall./ materials. Add-on possible.	Glaze	19:40
6	IPS e.max ZirCAD Superspeed sintering Prime	Program for the fast sintering of a maximum of three crowns made from IPS e.max ZirCAD Prime ^[1]	Off ^[2]	15:00 ^[3]
7	IPS e.max ZirCAD Sintering Prime	Program for the sintering of bridges or a maximum of three crowns made from IPS e.max ZirCAD Prime ^[1]	Off ^[2]	23:00 ^[3]
8	IPS e.max ZirCAD Corrective/Stain/Glaze firing with IPS e.max CAD Crystall./	Program for the Corrective/Stain/Glaze firing of a maximum of three crowns or one 3-unit bridge made from IPS e.max ZirCAD with IPS e.max CAD Crystall./ materials. Add-on possible after Glaze firing.	Glaze	14:50
9	IPS Empress CAD Corrective/Stain/Glaze firing with IPS e.max CAD Crystall./	Program for the Corrective/Stain/Glaze firing of a maximum of four single-tooth restorations made from IPS Empress CAD with IPS e.max CAD Crystall./ materials.	Glaze	19:40
10	IPS e.max ZirCAD Speed sintering MT Multi	Program for the fast sintering of a maximum of three crowns made from IPS e.max ZirCAD MT Multi ^[1] .	ZrO ₂ wet ^[2]	55:20 ^[3]
11	IPS e.max ZirCAD Sintering MT Multi	Program for the sintering of a maximum of three crowns or a three-unit bridge made from IPS e.max ZirCAD MT Multi ^[1] with higher translucency.	ZrO ₂ wet ^[2]	69:00 ^[3]
12	IPS e.max ZirCAD Speed sintering LT	Program for the fast sintering of a maximum of three crowns or a three-unit bridge made from IPS e.max ZirCAD LT ^[1] .	ZrO ₂ wet ^[2]	22:00 ^[3]
13	IPS e.max ZirCAD Sintering LT	Program for the sintering of a maximum of three crowns or a three-unit bridge made from IPS e.max ZirCAD LT ^[1] with higher translucency.	ZrO ₂ wet ^[2]	29:50 ^[3]

^[1] This sinter program may only be used for restorations made from blocks.

^[2] Predrying may be deactivated for dry-milled zirconia.

^[3] Program duration without predrying

^[4] R&D Ivoclar, Schaan

Important: Predrying is mandatory for crystallization and glaze programs.

For more detailed information, please refer to the Instructions for Use of the respective material.

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