# VOCO

## Admira® Fusion Flow

In accordance with DIN EN ISO 4049

### Instructions for use

#### Product description:

**Admira Fusion Flow** is a flowable, light-curing, radiopaque nanohybrid ORMOCEP® restorative material. **Admira Fusion Flow** contains 74 % w/w inorganic fillers and is used with a dentine-enamel bond.

#### Shades:

A1, A2, A3, A3.5, A4, B1, B2, C2, Incisal, BL, OA2, WO

#### Indications:

- Filling of small cavities and extended fissure sealing
- Blocking out undercuts
- Lining or coating cavities
- Fillings of class III - V cavities
- Repairing fillings, veneers and temporary restorations
- Luting translucent prosthetic pieces
- Locking, splinting of loose teeth

#### Preparation:

Clean the teeth to be treated. If required mark contact points. Allow the material to reach room temperature before use.

#### Shade selection:

To select a shade, compare the enclosed shade guide with the cleaned tooth before anaesthesia. Moisten the **Admira Fusion** shade sample and tooth and select the shade, preferably in daylight.

#### Cavity preparation:

In principle, cavity preparation should be carried out according to the rules of adhesive filling therapy and should be minimally invasive to conserve healthy tooth substance. Bevel the enamel margins on anterior. Clean and dry the cavity afterwards. Non-carious cervical lesions do not have to be prepared; thorough cleaning is sufficient here.

#### Creating a dry working field:

Ensure that the work area is sufficiently dry. Use of a rubber dam is recommended.

#### Matrix placement:

Take appropriate measures for areas where the applied **Admira Fusion Flow** can flow out. Translucent matrices that are wedged in the approximal area are advantageous. Minimal separation facilitates the shaping of the approximal contact and insertion of the matrix.

#### Pulp protection:

A suitable pulp-protective liner that can be covered with a stable cement, if necessary, should be applied in close proximity to the pulp.

#### Bond material:

**Admira Fusion Flow** is used in the adhesive technique with a dentine-enamel bond. All light-curing bond materials can be used. Follow the respective instructions for use with regard to preparation (etching technique) and application.

#### Application of Admira Fusion Flow:

Lock the enclosed application tip on the syringe with a clockwise turn and directly apply **Admira Fusion Flow**. For fillings with a thickness of over 2 mm, apply and cure in layers. Do not pull the plunger back on the syringe during or after use to guarantee the function of the non-dripping syringe.

#### Light-curing:

Conventional polymerisation devices are suitable for light-curing the material. The curing time is as follows when using an LED-/halogen light with a minimum light-output of 500 mW/cm<sup>2</sup>:

- 20 s for shades A1, A2, A3, A3.5, B1, B2, C2, Incisal, BL
- 40 s for shades A4, OA2, WO

Hold the light emission tip of the device as close as possible to the surface of the filling. Otherwise, the curing depth may be reduced. Incomplete curing may lead to discolouration and discomfort.

#### Finishing:

The restoration can be finished and polished immediately after removing the matrices (e.g., fine or extra-fine diamond grinder, polisher). The tooth should be fluoridated as a final step.

#### Information/precautionary measures:

- **Admira Fusion Flow** contains methacrylates and BHT. **Admira Fusion Flow** should not be used in case of known hypersensitivities (allergies) to any of these ingredients.
- The enclosed application tips are intended for a single use and must be disposed of after only one use. Return the cap to the syringe after use.
- Phenolic substances, especially preparations containing eugenol or thymol, interfere with the curing of filling composites. Avoid the use of zinc oxide eugenol cements or other eugenol-containing materials in combination with filling composites.

#### Storage:

Store at temperatures of 4 °C to 23 °C. To avoid exposure to light and potentially resulting polymerisation, syringes should be closed immediately after dispensing. Do not use after expiry date.

Our preparations have been developed for use in dentistry. As far as the application of the products delivered by us is concerned, our verbal and/or written information has been given to the best of our knowledge and without obligation. Our information and/or advice do not relieve you from examining the materials delivered by us as to their suitability for the intended purposes of application. As the application of our preparations is beyond our control, the user is fully responsible for the application. Of course, we guarantee the quality of our preparations in accordance with the existing standards and corresponding to the conditions as stipulated in our general terms of sale and delivery.

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VC 60 AA2816 E1 1115 99 © by Voco
ORMOCER® is a registered trademark of Fraunhofer Gesellschaft zur Förderung der angewandten Forschung e.V., München, Germany.
VOCO GmbH