RETRAKTIONSFADEN

Stand: 150224
2024-REV-004

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Patientenzielgruppe

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en Instructions for use

Patient target group

For the treatment of women and men of all ages as well as children.

User

For dental use only.

Description

Knitted retraction threads made of 100% cotton.

Indications

For pushing aside and retracting the gingivae.

Contraindications

Use on inflamed gums is to be avoided.

Clinical benefit

For preparing an impression for the fabrication of crowns or bridges. For deep fillings or cervical fillings.

Performance characteristics of the product

Numerous tiny loops form an interlocking chain. This unusual design results in even, lasting pressure on the gum line. Due to its special knitted construction, M+W Retraktionsfäden ensures excellent tissue displacement and is better retained in the sulcus than conventional retraction threads. M+W Retraktionsfäden thread also offer excellent

haemostatic properties, as they absorb significantly greater amounts of haemostatic agents than conventional threads.

- Size #2 and occasionally size #3 are suitable for molars.

- Size #1 is often suitable for premolars and upper teeth.

- Sizes #0 or #00 are often used for the lower front teeth.

- The thread size #000 is used for preparations or cementing of veneers and for thin, tightly fitting sulcus tissue. This size is also used as the lower thread for the double-thread technique. The nature of the tissue and the desired size of the free space may make it necessary to use sizes other than those described above.

Specific precautions

Due to the type of packaging used, the retraction threads are pressed. Please select one size larger than expected.

Application

1. Select the appropriate size of retraction thread.

2. Use a thread stopper to insert the retraction thread into the sulcus.

3. After inserting the retraction thread, rinse the affected areas. An optimal displacement of the tissue will take place within 1-3 minutes.

4. Remove the retraction thread from the sulcus before taking an impression. Lightly spray with water to clean the preparation and test the effect of the haemostatic agent.

Warnings / residual risks

None

Storage

Dry and at room temperature. The Product should be stored between 0° C - 22° C.

Minimum shelf life

4 years if stored properly. See outer packaging.

Variants

The different sizes of retraction cords

REF 300 270, size #000

- Preparation and cementation of veneers

- Restorative treatment for thin and brittle tissue

REF 300 271, size #00 / **REF 300 272**, size #0

- Lower front teeth

- For luting veneers close to gingival or subgingival areas

- Restorations in classes III, IV + V

- Second thread for "two-thread" technique

REF 300 273, size #1

- Sizes 1 and 2 are particularly effective for tissue control and/or restraining if they are previously immersed in a haemostatic solution before or after crown preparations

- Use a protective „pre-preparation thread“ for front teeth

REF 300 274, size #2

- Upper thread for "two-thread" technique

- Protective „pre-preparation thread“ for anterior teeth

Disposal

Dispose of properly in accordance with the local official regulations. Non-contaminated packaging can be sent to a recycling facility.

Side effects, reactions or other undesired occurrences for the patient and/or user (even in cases of doubt) must be reported to the manufacturer promptly and in detail.

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M+W Dental Müller & Weygandt GmbH · Reichardsweide 40 · 63654 Büdingen/Germany · Tel.: +49 (0) 60 42 - 88 00 88