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Important note regarding use

Please read these instructions carefully and in their entirety before using the product for the first time. The product must only be used as described in these instructions. Do not use the product for any purpose other than that specified. Please follow all safety precautions and instructions contained herein. The product must only be used by persons who have read and understood these instructions.

We assume no liability for damage resulting from improper use, mishandling, or failure to follow these instructions. This does not affect mandatory liability under the Product Liability Act, liability for injury to life, limb, or health, or liability arising from willful misconduct or gross negligence.

Material description

DuoSep is a separating agent for the insulation of equal and non-compatible materials which is to be applied by means of a brush.

Field of application

A-silicone / A-silicone
C-silicone / A-silicone
Polyether / A-silicone
Polyether / polyurethane

Processing

Shake before using the material. Apply equally onto a clean and dry contact surface. Avoid the development of puddles! Let the material dry well. The dry material creates a very thin insulation layer which guarantees a safe separating film.

Important advices

- DuoSep technical filling reasons DuoSep can easily settle when storing. For this reason always shake well before use.
- DuoSep can easily be removed with water.
- For dental use only.

Storage conditions

At a room temperature, i. e. 18–28 °C

Shelf life: 2 years in the closed original container

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