simplee Scaler Tip

Stannt:181219

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simplee Scaler Tip

GB Indications for Use

Scaler tips work with ultrasonic scaler machines. The ultrasonic energy is created by the scaler machine and converted into vibrations by the hand-piece; these vibrations are then focused and applied to the area to be treated by the Scaler Tip. Ultrasonic Scaler Machines and Tips are only to be used by qualified dental professionals.

General (Scaling –prophylaxis)

These are used for the removal of supra-gingival stains and deposits. The power setting used should be the lowest effective power according to the instructions of the scaler machine.

Periodontal

These are used for sub-gingival work. The power settings will depend upon the tip used and the operation to be carried out; in general though, thinner tips require lower power.

General Instructions.

1) Contraindications. Follow scaler machine instructions for contraindications.

2) Scaler machine's instructions. Please read and follow your scaler machine's instructions.

3) Cleaning and sterilization.

Ultrasonic tips must be sterilized before first use and after each use (they are not delivered sterile).

Cleaning

The removal of organic deposits is very important and will achieve best results by putting them immediately after use into 70% Isopropyl alcohol (Isopropanol) or a low-alcohol cleaning solution. After that use an ultrasonic bath with a low-foaming detergent or thermal disinfection. Alternatively, you can clean them in warm water with a lowfoaming detergent and an autoclavable brush.

Attention

Use only chemical disinfectants that are pH neutral and chlorine free to avoid damage to the ultrasonic tips (e. g. M+W Instrument bath plus).

After Cleaning

Rinse the cleaned tips thoroughly in warm water to remove all chemicals and let them dry completely.

Simplee ultrasonic tips are made out of stainless steel of the highest quality.

Therefore, the use of an antirust agent before steam sterilization is acceptable, although it is not required. Pack them in suitable packaging, bags, trays or cassettes.

Attention

Do not use closed containers during sterilization with steam or ethylene oxide. The cleaning and Disinfecting device must be approved by its manufacturer for cleaning and disinfection be released by dental instruments and EN ISO 15883-1.

Sterilize with ethylene oxide

Put the dried packaged materials in the unit and work according to the manufacturer's instructions.

Regarding sterilization

Steam autoclaving is the preferred method of dentistry. DO NOT use chemiclaves or drying ovens for sterilizing Simplee ultrasonic tips.

The scaling tip has no sterilization cycle limit, but after the tip is worn exceeding 2mm, the scaling tip needs to be replaced.

4) Tip Selection. Standard tips are best for medium to heavy deposits. Universal tips are designed for use in pockets less than 4mm.

5) Tip Attachment. Tips should be loaded onto the hand-piece using the appropriate Torque Wrench.

6) Power Setting. When the scaler tip is used with the hand-piece for the first time, set the scaler machine to the minimum power setting, and then adjust to the correct power, according to the machine's instructions.

The use of excessive power may cause patient discomfort and may cause micro fractures in the tip;

7) Cooling. In order to cool the scaler tip in use, the power and then the water flow need to be set, follow machine instructions for this;

8) Tip Modification. DO NOT modify, sharpen or bend the tip in any way; this will result in incorrect ultrasonics and, will damage (or possibly break) the tip and the scaler machine.

9) Damaged Tips. If tip is bent out of shape, if a tip has been dropped, or if a change of power is experienced during use, discard and replace.

10) Storage Environment. Scaler tips should be autoclaved after use. Scaler tips should be stored in a dry environment with humidity lower than 70%.

11) Tip Life. Scaler tips effectiveness is reduced by wear; once the tip has worn by 2mm, it will no longer be effective. As a guide, basic scaling tips should last for about 100 uses, with Endo tips needing much more frequent change; thinner tips have shorter life. Some of the specialist tips, such as those with diamond or Titanium Nitride coats will be replaced after just 3 or 4 uses.

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