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Silicone polishers

1. Warning notices

- Strong acids or strong bases may lead to corrosion of the stainless steel shaft.
- Rinse the polisher with distilled water after the treatment with cleaning and disinfectant solutions.
- Avoid temperatures > 150° C.
- Ultrasonic bath must not exceed temperatures of 42° C because of the possible coagulation of proteins.

2. Restriction of reprocessing

- Disposable products delivered unsterile, marked with the symbol  may only go through the validated sterilisation cycle once before initial use.

3. Comment

- Concerns all rotating polishing and grinding instruments that are classified as semi-critical according to RKI-guideline.
- All instruments are delivered unsterile and must be run through the indicated cycle before and after each use.
- The label on the cleaning and/or disinfecting solution must specifically say „suitable for rubber polishers or synthetics/silicones“.

4. Instructions

- The instruments must be pre-cleaned under running water with a brush (plastic) directly after use.
- The manual cleaning is done under running water with a brush (plastic brittles).
- The ultrasonic assisted cleaning must be done with a suitable agent and disinfectant.

The instruments must be rinsed afterwards under running water.

- When cleaning with a thermal disinfectant the manufacturer's specification according to DIN EN ISO 15883 as well as the cleaning programme as indicated by the manufacturer in the operating instructions must be complied with.
- For disinfection a solution has to be used that is classified as suitable for rubber and silicone polishers and synthetics by the disinfectant manufacturer. Exposure times and concentrations recommended by the manufacturer must be adhered to.
- After treatment with cleaning and disinfectant solutions, the polishers have to be rinsed with distilled water.
- Dry with fresh, clean, lint-free cellulose tissues.
- For maintenance a visual check of all instruments has to be conducted with optical magnification (5-10 fold).
- No residues -> continue to sterilisation.
- Visible residues -> repeat cleaning. Reject and dispose of instruments in the event of discernible defects.
- Sterilisation: For all instruments that need to be sterilized in accordance with EN ISO 17664 and all national valid legal requirements.
- Steam sterilization: Appliance according to EN 13060, validated procedure. Category S- or B-steriliser.
- Holding time: Full cycle at least 5 min
- Sterilisation temperature: 134° C
- Drying time: 10 min
- Threshold values of contents for feed-water and steam condensates

- Loading of steriliser according to manufacturer's instructions
- Follow manufacturer's operating instructions
- Store instruments packed and protected from recontamination in proven suitable sterile packaging, cassettes or retainers.
- Repetitive reprocessing can change both the look and feel of the instruments slightly, but does not interfere with the instrument's function.

5. Validation conditions

Manual cleaning:

- Cleaning and disinfectant solution Dürr Dental ID 212
- Concentration: 2%
- Holding time: 5 min

Ultrasonic assisted cleaning:

- Dürr Dental ID 212
- Type of appliance: Ultrasonic
- Concentration: 2%
- Holding time: 2 min

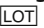
Automatic cleaning:

- Cleaning and disinfecting machine Miele G 7883
- Program: SPECIAL 93° C – 10'
- Holding time: 10 min
- Detergent: Dr. Weigert – neodisher MediClean Dental
- Rinsing: 3 min at 75° C with neodisher Z Dental

Sterilisation:

- Holding time full cycle: 5 min
- Sterilisation temperature: 134° C
- Drying time: 10 min
- Threshold values of contents for feed-water and steam condensates

Marking:

The  number is shown on the package.

Date of issue:

02-2020

Only supplied to dentists and dental laboratories or upon their instructions.

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Coltene/Whaledent AG ■