**3M ESPE****Ketac™ Cem radiopaque**

Glass Ionomer Luting Cement

Glasionomer-Befestigungszement

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**Indication**

• Cementation of inlays, crowns, and bridges made of metal, with metal frameworks, or of high-strength ceramic or composite, provided they are suitable for conventional cementation.

• Cementation of crowns and bridges made of metal, with metal frameworks, made of high-strength ceramic or composite, provided they are suitable for conventional cementation.

• Cementation of posts and cores, made of metal or high-strength ceramic, provided they are suitable for conventional cementation.

• Cementation of orthodontic brackets.

**Precautionary Measures**3M MSDs can be obtained from [www.3mm.com](http://www.3mm.com) or contact your local subsidiary.**Preparation**

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• The preparation should be dry so that the surface has a little shiny appearance. Excessive heating with compressed air can negatively affect adhesion or result in post-operative sensitivity after sealing the restoration.

• The surfaces of the restoration should be dried with compressed air, avoid further contamination by saliva.

**Pulp Protection**

The glass ionomer cement must not be applied directly to dentin – e.g., on inlays – situated close to the pulp or to the exposed pulp.

• Prior to using the impression, cover areas close to the pulp using a hand-setting calcium hydroxide preparation.

**Disposal**

• Sprühe bottle in bottle powder.

• Screw off cap and replace by dispensing dropper.

• For cementing water, the recommended powder-to-liquid mixing ratio is 3:1 – with a dry time of 10 minutes.

• Add 10 drops of water on plastic, ceramic, acrylic.

• Dust powder and liquid rest until each other on a mixing pad or glass slab.

• Keep both upright until the dispensing. Crystallized liquid must not be allowed to form on the dispensing dropper.

• Clean dropper after use.

• Clean dropper to smooth the paste until a homogeneous mixture is obtained.

**Application**

During application, water and saliva must be prevented from entering the working area.

• Apply a thin coat of cement to the surface of the restoration to be cemented and to its preparation.

• Subsequently set the restoration.

• Avoid overheating the restoration. Applying hydrostatic pressure build up in tightly fitting restorations may cause pulp discomfort.

**Times**

The following time scale applies at a room temperature of 23°C/73°F and 50% relative atmospheric humidity

min sec

Mixing 1:00

Application incl. mixing 3:30

Setting from begin of mix 7:00

Higher temperatures shorten the working time, whereas lower temperatures will extend it period (e.g., setting on a cold desk slow).

A higher quantity of powder should be added to the liquid to produce viscous consistency.

The quantity of cement required for a crown should just about drip from the spatula.

• Continue to smooth the paste until a homogeneous mixture is obtained.

**After Curing**

• Gently clean off the surface of the treated tooth and the surrounding areas, remove any resinous remains still remaining. This is especially important when cementing to implant abutments.

**Removal of Excess**

• Remove excess using a Heidemann spatula and/or probe 7 min, after beginning of mix.

**After Cleaning**

• Gently clean off the surface of the treated tooth and the surrounding areas, remove any resinous remains still remaining. This is especially important when cementing to implant abutments.

**Storage and Shelf Life**

Keep the powder in a dry place away from damp.

Store the product at 15–25°C/59–77°F.

Do not use after the expiry date.

**Customer Information**

No person is authorized to provide any information which deviates from the information provided in this instruction sheet.

**Warranty**

3M Deutschland GmbH warrants this product will be free from defects in material and manufacture.

3M Deutschland GmbH MAKES NO OTHER WARRANTIES.

NO WARRANTY IS PROVIDED AS TO THE MERCHANTABILITY OR FITNESS FOR A PARTICULAR PURPOSE. User is responsible for determining the suitability of the product for user's own application. If this product is defective within the warranty period, your exclusive remedy and 3M Deutschland GmbH sole obligation shall be repair or replacement of the 3M Deutschland GmbH product.

Limitation of Liability

Except where prohibited by law, 3M Deutschland GmbH will not be liable for any loss or damage arising from this product, whether direct, indirect, special,

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