



ALGICIT ADHÄSIV

Inhalt/Content: e 15 ml

Haftvermittler für Alginat

Das Adhäsiv-Liquid erzeugt eine starke, sehr dünne und permanente Klebeschicht für Alginat.

Anwendung: Mit dem Pinsel den Abformlöffel bestreichen und nach einer kurzen Trocknungszeit (ca. 5 Min.) das Abformmaterial aufbringen. Klebstoffreste können von Metalllöffeln mit einem Adhäsiv Löser entfernt werden. Enthält Aminoethylethanolamin. Kann allergische Reaktionen hervorrufen.

 ALGICIT ADHESIVE Adhesive for Alginates

The Adhesive Liquid creates a strong, very thin and permanent layer of adhesive bonding for alginate impression material.

Application: Use the brush to apply a thin layer of the adhesive to the tray. Allow to dry completely (at least 5 min.). Use adhesive removal to remove any adhesive residue from metal trays. Contains aminoethyl ethanolamin. May produce an allergic reaction.



LOT

siehe Aufdruck



Stand: 23.10.17

MW Dental



Müller & Weygandt GmbH • Reichardsweide 40 • D-63654 Büdingen • Tel.: +49 (0)6042 – 88 00 88