**REF 400 105**

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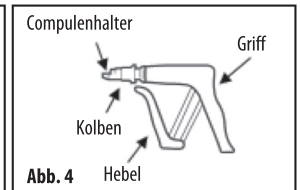
Abb. 4

## Ⓧ DIRECTIONS FOR USE

**Carefully read the instruction before using this product.**

### CAPSULE LOADING

Insert and secure the compule in compule holder (Fig. 1).

**Please Note:** The compule cannot be inserted if the plunger is extended. Retract the plunger by releasing the trigger.

### EXTRUDING THE MATERIAL:

Direct the compule nozzle into the prepared surfaced of the tooth. Extrude the required amount of composite into the prepared cavity, after use release the pressure on the compule by reducing pressure on the lever. Compules are single use device. Cure the extruded material following the manufactures instructions. (Fig.2).

### REMOVAL OF THE CAPSULE:

Remove the used compule from the M+W Composite Gun by twisting clockwise and pulling upwards (Fig.3).

### CLEANING:

Clean the M+W Composite Gun with a gauze soaked in alcohol.

### STERILIZATION:

M+W Composite Gun can be autoclaved at 121–134°C (250–273°F) for 10 to 30 minutes (excluding heat-up time). Please follow manufacturer's instructions.

**Please Note:** The use of chemical disinfectants/sterilants may cause damage to the applicator and are therefore not recommended.

### WARNING:

1. Prior to use, sterilize the M+W Composite Gun.
2. Do not use the M+W Composite Gun on patients who have a history of allergy or show signs of allergy to metals. If any such reactions are experienced, discontinue the use of the product and refer to a physician.

3. An operator who has a history of allergy to metals should not handle the M+W Composite Gun. In case of allergy, discontinue use and consult a physician.

4. Take care not to get your hand or fingers caught between the lever and the grip.

5. This product should only be used by a dental professional.

6. This product should only be used for the applications described herein.

### BATCH CODE:

The batch code gives an open date of manufacture in month, year, day format with a numerical suffix to uniquely identify the batch of material. Please quote this batch number in all correspondence.

This product has been formulated for use in dentistry and is intended for use by dental professionals only.

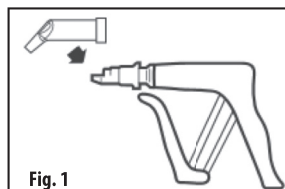
Fig. 1

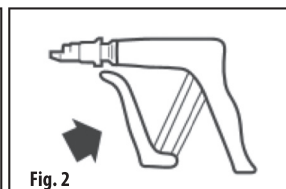
Fig. 2

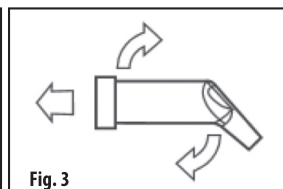
Fig. 3

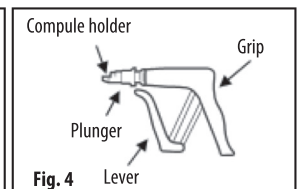
Fig. 4

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