Bluephase® G4

Start-up

- 1 ✓ Unpack handpiece
✓ Detach the light guide by pulling it out
✓ Clean them (See Instructions for Use)
- 2 ✓ Unpack charging base
✓ Slide connector plug into its socket
- 3 ✓ Connect the mains cable to the power supply unit and supply mains

- 4 ✓ Slide battery straight into the handpiece until you hear and feel it click into place.
- 5 ✓ Place handpiece in the corresponding rest in the charging base


i Disposable protective sleeves are recommended for each patient. Attach anti-glare cone or anti-glare shield to the light guide.

i If the battery is fully discharged, the charging time is approx. 2 hours

6 Operation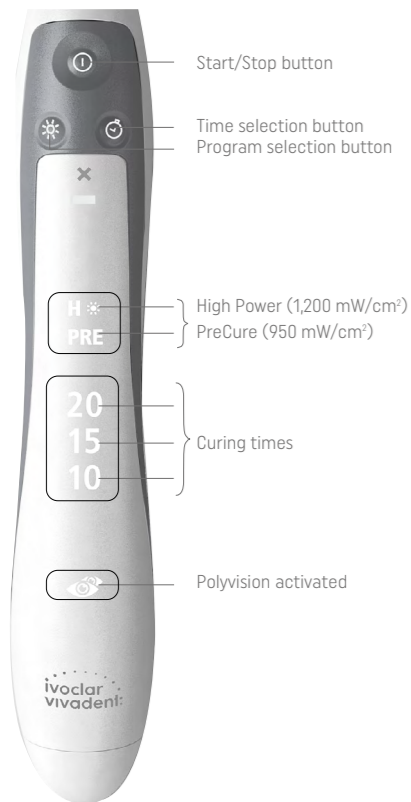

Curing times		HIGH POWER 1,200 mW/cm ²
Restorative materials	Composites • 2 mm ¹⁾ IPS Empress Direct / IPS Empress Direct Flow Tetric EvoCeram / Tetric EvoFlow Tetric / Tetric Basic White	10 seconds
	• 4 mm ²⁾ Tetric PowerFill / Tetric PowerFlow Tetric Basic White	10 seconds
	Heliomolar / Heliomolar HB / Heliomolar Flow All conventional composites ³⁾	15 seconds
Indirect restorations / cementation materials	Compomer ³⁾ Compoglass F / Compoglass Flow	20 seconds
	Variolink Esthetic LC ⁴⁾ / Variolink Esthetic DC ⁵⁾ Multilink Automix ⁵⁾ / SpeedCEM Plus ⁵⁾	per mm ceramic: 10 seconds per surface 20 seconds per surface
Adhesives	Adhese Universal / Excite F / Excite F DSC Heliobond / Syntac	10 seconds
Temporary materials	Telio CS Link Telio CS Inlay/Onlay Telio Stains Telio Add-On Flow	10 seconds per surface 10 seconds 10 seconds 15 seconds
	Heliocore F Plus	10 seconds
Miscellaneous	IPS Empress Direct Color IPS Empress Direct Opaque MultiCore Flow / Multicore HB	20 seconds 20 seconds 20 seconds

1) Applies to a maximum layer thickness of 2 mm and provided that the Instructions for Use of the respective material do not state any other recommendation (might be the case e.g. with dentin shades)
2) Applies to a maximum layer thickness of 4 mm and provided that the Instructions for Use of the respective material do not state any other recommendation (might be the case e.g. with dentin shades)
3) Applies to a maximum layer thickness of 3 mm
4) Applies to light-curing
5) Applies to dual-curing

What if ...?

If any of these errors persist after trying their corresponding rectification, please contact your dealer or your local Service Centre.

Indicator	Causes	Rectification
RED "X" LIGHTS UP 	The device is overheated	Allow the device to cool down and try again after certain time.
	Electronic component of the handpiece is defective.	Remove and reinsert the battery.
RED "X" AND BATTERY SYMBOL LIGHT UP 	Battery empty	Place the device in the charging base and charge it.
	Battery contacts dirty	Remove battery and clean the battery contacts.
BATTERY IS NOT LOADING	Charging base does not recognise the battery	Remove the handpiece from the charging base and place it back again
	Battery empty	Remove and reinsert the battery and place the handpiece in the charging base
	Electronic component is defect	Please contact your dealer or your local Service Centre

- Repeated consecutive curing cycles (more than 2 curing cycles in succession) on the same tooth entail the risk that the pulp may become damaged due to the increase in temperature.
- The PreCure program must not be used for conventional light-curing!
- Attention: When Polyvision is activated, the intelligent glare protection is automatically active!

For dental use only!
Made in Austria

Manufacturer
Ivoclar Vivadent AG
9494 Schaan/Liechtenstein
www.ivoclarvivadent.com

Rx ONLY
In USA distributed by
Ivoclar Vivadent Inc.,
175 Pineview Drive,
Amherst, NY 14228, USA

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