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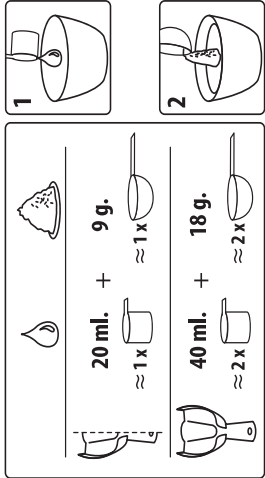
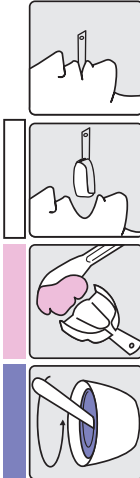
(*) Measurements taken at 23°C with deionised water.

(*) mixing time 45"

(*) working time 1'45"

(*) initial setting time 1'55"

Nettogewicht: 500 g

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only 30" in mouth

Ⓢ M+W Algicrit Dimension Color does not require special treatment before use. Dust-free material. However, caution is advised when opening the bag and withdrawing the powder.

While mains water may be used, the de- clared times were measured by mixing the powder with deionised water at 23°C.

Vigorously mix the powder with water.

When using specific mechanical mixers for alginatc-based impression materials, fol- low the manufacturer's instructions.

Follow the chromatic phase indica- tions which ensure that the product is mixed perfectly and inserted into mouth at the right moment.

Variations in temperature and/or water hardness and the speed and/or friction

during mixing cause small changes in the working time and in the initial setting time, which are automatically indicated by the colour changes.

Wash, rinse and shake the impression and pour the model. The model can also be poured within up to 100 hours with excel- lent results. The impression must be stored in a sealed plastic bag without adding any- thing else. **Do not leave the impression exposed to air or immersed in water.**

Keep the container tightly closed in a cool, dry place.

Net weight: 500 g.

Net volume after mixing: 1400 ml.

MADE IN ITALY