VOCO**Bifluorid 10®****GB****MD** EU Medical device**CE** 0482

Fluoride varnish for the treatment of hypersensitivity

Instructions for use

Product description:

Bifluorid 10 is a fast-drying dental suspension that adheres well to dry enamel and dentine. Thus, it is possible to seal the dentine tubules tightly for an extended period of time.

Bifluorid 10 forms a waterproof, protective coat that insulates against thermal and chemical influences.

1 g **Bifluorid 10** contains:

- 50 mg sodium fluoride (equals 23 mg fluoride) and
- 50 mg calcium fluoride (equals 24 mg fluoride)

Indications:

- Treatment of all hypersensitivities (dentine hypersensitivity) including:
 - hypersensitivities of cervical areas and crown margins
 - hypersensitivity after professional cleaning and calculus removal
 - hypersensitivity of tooth surfaces after preparation and / or grinding
 - hypersensitivity due to worn occlusal surfaces and teeth carrying clasps
 - hypersensitivity after injury of the enamel (e.g. fractures, chipping)
 - sealing of cavity margins after restoration, especially after the etching technique has been employed
 - as dentine protection/lining under amalgam fillings

Contraindications:

Bifluorid 10 contains eugenol. **Bifluorid 10** should therefore not be used in patients with a known hypersensitivity (allergy) to this ingredient.

Patient target group:

Bifluorid 10 is suitable for use in all patients without any age or gender restrictions.

Performance features:

The product's performance features satisfy the requirements of the intended use and the relevant product standards.

User:

Bifluorid 10 should only be used by a professionally trained dental practitioner.

Use:

- Preparation:**
Clean and thoroughly air-dry surfaces to be treated.

2. Dispensing

- Bottle**
Shake **Bifluorid 10** bottle well before each use. The solid particles must be well dispersed. The space in the bottle and two metal balls facilitate homogeneous resuspension.
Dispense drops of **Bifluorid 10** onto an applicator or preferably a **Pele Tim** foam pellet. Tightly close **Bifluorid 10** bottle after use.
- SingleDose**
Detach a **SingleDose** application at the perforation and turn the printed side up. Pierce the foil with a **Micro Tim**, enlarge the opening and wet the brush with circling movements.

3. Application

Thinly coat the surfaces. Caution: Layers that are too thick may easily peel. Allow the fluoride varnish to be absorbed for 10 - 20 s, then dry with air. **Bifluorid 10** remains on the applied surfaces for several days if it is correctly applied and appropriate oral hygiene is carried out (**teeth should not be brushed for 12 - 24 h after application**).

For hypersensitive cervical areas, crown margins or similar indications, repeat treatment two to three times at intervals of 7 days as appropriate. Repeated application should only involve the drying of surfaces and reapplication of the varnish.

Warnings, precautionary measures:

- Bifluorid 10** should only be used with extreme caution in individuals who may not have full control over their swallowing reflex (e.g. preschool children and the disabled).
- Systemic fluoride therapy (e.g. fluoride tablets or fluoridated table salt) should be discontinued for several days following application of **Bifluorid 10**.
- Incorrect application (see section entitled "Method of use") may result in major amounts of fluoride being introduced into the oral cavity. Swallowing such an amount may lead to nausea, vomiting and / or diarrhoea. Routinely exceeding a total daily amount of 2 mg of fluoride (systemic and local) during odontogenesis can result in disturbances in the mineralisation of the enamel. This disturbance, also referred to as dental fluorosis, manifests itself in the appearance of white spots on the enamel.
- The viscosity of **Bifluorid 10** must be low enough to allow the liquid to easily drip from the dropper after being shaken
- The **solvent** should also be used for cleaning the instruments etc.
- Our information and/or advice do not relieve you of the obligation of checking that the products supplied by us are suitable for their intended purpose.

Storage:

Protect **Bifluorid 10** from direct sunlight and store at 4 °C - 23 °C. Do not use after the expiry date.

Disposal:

Dispose of the product in accordance with local regulations.

Reporting obligation:

Serious events such as death, temporary or permanent serious deterioration of a patient's, user's or other person's health condition and a serious risk to public health that arise or could have arisen in association with the use of **Bifluorid 10** must be reported to VOCO GmbH and the responsible authority.

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VC 60 AA1616 E1 0221 99 © by VOCO