



SIOPLAST SILIKONADHÄSIV

Stand: 23.10.17

REF 200 280

Inhalt/Content: e 10 ml

Haftvermittler für A- und C-Silikone

Das Adhäsiv-Liquid erzeugt eine starke, sehr dünne und permanente Klebeschicht für Silikon-Abformmaterialien.

Anwendung: Mit dem Pinsel den Abformlöffel bestreichen und nach einer kurzen Trocknungszeit (ca. 5 Min.) das Abformmaterial aufbringen. Klebstoffreste können von Metalllöffeln mit einem Adhäsiv Löser entfernt werden.

Sicherheitshinweise: H361 Kann vermutlich die Fruchtbarkeit beeinträchtigen oder das Kind im Mutterleib schädigen. Behälter dicht verschlossen halten.

CE SIOPLAST SILICONE ADHESIVE Adhesive Agent for A and C Silicone

The adhesive liquid produces a strong, very thin and permanent adhesive layer for silicone formable materials.

Application: use the brush to coat the impression tray and apply the material to be formed after a short drying time (approx. 5 min). A metal spoon and an adhesive solvent can be used to remove any adhesive residue.

Safety instructions: H361 Suspected of damaging fertility or the unborn child. Keep container tightly closed.

Gefahr/Danger

Enthält/Content: Toluol, n-Hexan



LOT

siehe Aufdruck



M+W Dental



Müller & Weygandt GmbH • Reichardsweide 40 • D-63654 Büdingen • Tel.: +49 (0)6042 – 88 00 88