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# EVACUATION CANNULAE COLOR/ EVACUATION CANNULAE COLOR KIDS

## **(GB)** Processing instructions

### Information on reprocessing according to DIN EN ISO 17664

#### Non sterile, reprocessing and sterilize before use!

#### 1. Application and use

The M+W suction cannula fits handpieces with hose fittings according to DIN 13938 (ø 16 mm).

The Dental Central suction cannula is not suitable for fitting to a non-standardised outer cone, such as e.g. Siemens Siro 1 to V or Adec. This may cause severe damage to the M+W suction cannula.

#### 2. Product life:

The M+W suction cannula is made of high-quality plastic. It is autoclavable. The end of the product life is generally determined by the results of wear and tear. Frequent reprocessing does not affect the performance of the M+W suction cannula.

Based on standard use, the M+W suction cannula can be resterilised 100 times (standard value) in the instrument autoclave.

#### 3. Reprocessing

##### 3.1 General information on reprocessing

The M+W suction cannula has an easy-to-clean surface with no sharp edges and corners. We recommend a commercially available disinfectant for cleaning and disinfecting. **Complete moistening** of the M+W suction cannula is essential during the disinfection process. Compliance with the current legal requirements for reprocessing medical devices has to be ensured (see e.g., www.rki.de). Please observe the information and instructions for use of the manufacturer of the cleaning / disinfection unit and the manufacturer of the cleaning / disinfectant agent.

The M+W suction cannula can be sterilised in the autoclave at a maximum temperature of **134° C** without problem as long as **any metal contact is avoided**.

##### Please observe the following instructions for the sterilisation process:

1. Line the sterilising trays with filter paper.
2. Do not place instruments on top of, or immediately adjacent to M+W suction cannulae.
3. The M+W suction cannulae should not be flush with the tray rim.
4. Do not slide trays loaded with M+W suction cannulae into tracks close to the heating rods (use middle track).
5. Avoid contact of the M+W suction cannulae with the inner surface of the autoclave because this heats up to much higher temperatures than the sterilisation chamber.
6. M+W suction cannulae should not be sterilised alone. To ensure a more even temperature distribution within the autoclave, it is recommended to sterilise instruments at the same time.

**Should, despite of the above-mentioned measures, a deformation of the M+W suction cannulae be observed, it is recommended to check the temperature of the autoclave.**

Latest revision: 21.04.2022

##### 3.2 Storage after use and transport to the processing site

Place the M+W suction cannulae immediately after use on the patient in an instrument tray filled with a suitable cleaning / disinfectant agent (alkaline, aldehyde-free). This soaking prevents drying of residues. Reprocessing of the M+W suction cannulae should be carried out no later than one hour after use. The transport of the M+W suction cannulae to the processing site should be carried out in a closed instrument tray.

##### 3.3 Cleaning and disinfection

According to the recommendations of the Robert Koch Institute (RKI), preference should be given to automated processing. Therefore, automated processing will be described first, followed by manual processing as an alternative option.

###### 3.3.1 Automated processing

Equipment:

- Cleaning / disinfection unit
- Alkaline cleaning/disinfectant agent
- Instrument stand

Method:

• Remove M+W suction cannulae from the instrument tray immediately before automated processing and clean thoroughly under running water so that no residues of the cleaning / disinfectant agent will be transferred into the cleaning / disinfection unit.

• Place M+W suction cannulae in a suitable instrument stand.

• Place the instrument stand into the cleaning / disinfection unit in such manner that the spray jet is directly aimed at the M+W suction cannulae.

• Fill cleaning/disinfectant agent into the cleaning / disinfection unit as directed by manufacturer of the agent and unit, respectively, and start the programme.

• After completion of the programme cycle, remove the M+W suction cannulae from the cleaning / disinfection unit and dry them preferably with compressed air according to RKI recommendations. With regards to instrument stands, ensure that difficult-to-reach areas are dried as well.

• Carry out visual control regarding integrity and cleanness with suitable magnifying device. If traces of contamination can be identified on the M+W suction cannula after reprocessing, the cleaning / disinfection process must be repeated until no more contamination is visible.

• Final thermal disinfection of the not packed M+W suction cannulae in the autoclave using suitable stands or sieve trays and observing the information provided under 3.1.

###### 3.3.2 Alternative manual processing

Equipment:

- Cleaning brush
- Cleaning / disinfectant agent (alkaline, aldehyde-free)
- Instrument tray

Method:

• Remove M+W suction cannula from the instrument tray and thoroughly rinse clean under running water. Completely remove adherent contaminations with the cleaning brush, both internal and externally.

• Rinse M+W suction cannula under running water.

• Place M+W suction cannula in instrument tray filled with a suitable cleaning / disinfectant agent. Ensure complete moistening of the M+W suction cannula.

• Observe manufacturer's information regarding concentration and application time.

• At the end of the application time, rinse the M+W suction cannula thoroughly with water.

• Dry the M+W suction cannula preferably with compressed air according to RKI recommendations

• Carry out visual control regarding integrity and cleanness with suitable magnifying device. If traces of contamination can be identified on the M+W suction cannula after reprocessing, the cleaning / disinfection process must be repeated until no more contamination is visible.

• Final thermal disinfection of the not packed M+W suction cannulae in the autoclave using suitable stands or sieve trays and observing the information provided under 3.1.

##### 3.4 Control and function test

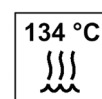
After reprocessing, the integrity of the M+W suction cannulae should be checked visually. In case of mechanical damage, including scratches, bite marks, porosity, tears and other signs of wear and tear, the M+W suction cannulae must be replaced by new ones.

##### 3.5 Transport and storage

Ensure that the M+W suction cannulae are transported and stored in a dust-free and humidityfree environment protected from recontamination.

##### 3.6 Final information

The manufacturer Angiokard Med. Spritzguss & Entwicklungstechnik GmbH ensures that the above-mentioned methods are suitable for reprocessing M+W suction cannulae. It is the processor's responsibility to ensure that the actual processing with the equipment, materials and staff used in the reprocessing facility will achieve the desired results. For this end, validation and routine monitoring of the procedure is usually required. Likewise, any deviation from the supplied instructions made by the processor should be carefully evaluated regarding their effectiveness and possible negative consequences.

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