KERAMIK ÄTZGEL

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Instructions for use:

- The M+W ceramic etch-gel is to be applied with a small disposable brush or plastic rod to the ceramic surfaces to be etched.

- Recommended is an etching time of 60 sec.

- After the application time, remove the acid residues entirely. To do this, intensively steam-blast the surface, spray it with water or clean it in an ultrasonic bath using distilled water. The etching of the ceramic surfaces is preferably carried out immediately before

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the insertion or after try-in. Avoid contamination of the etched surfaces (do not brush). However, if contamination occurs after etching, then the surface must be shortly etched again (approx. 10 sec). Dry well after etching, then continue to use adhesion promoters, etc. according to the manufacturers' instruction manual. Only for extraoral use! The M+W ceramic etch-gel must not be used for etching tooth substance in the mouth.

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Safety instructions:

H 310 Fatal in contact with skin. **H 331** Toxic if inhaled. **H 314** Causes severe skin burns and eye damage. Wear protective gloves/eye protection. Avoid breathing dust/fume/gas/mist/vapour/spray. **IF SWALLOWED:** Immediately call a **POISON CENTER** or doctor/physician. **IF IN EYES:** Rinse cautiously with water for several minutes. Remove contact lenses, if present and easy to do. Continue rinsing.

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