PONTIFORM AUTOMIX 2 / DIRECT MIX

Stand071019

D Gebrauchsinformation

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GB Instructions for use

M+W Pontiform automix 2/ direct mix is a bis-acrylic based provisional crown and bridge material that is peroxide free and methacrylate free. The tissue-friendly, low polymerization temperature prevents thermal damage to the pulp. M+W Pontiform automix 2/ direct mix incorporates a special snap-set curing characteristic and consequently has a long plastic working stage, during which the temporary restoration can be easily removed from the patient's mouth. M+W Pontiform automix 2/ direct mix is exceptionally fracture resistant and shade stable due to its outstanding mechanical properties and is therefore ideal for fabrication of long-term temporary restorations. M+W Pontiform automix 2/ direct mix guarantees a highly accurate fit of the finished temporary restoration with optimal marginal integrity. M+W Pontiform automix 2/ direct mix should be mixed in an automatic mixing unit.

Indication
Temporary crowns, bridges, inlays, onlays, partial crowns, veneers, long-term temporary restorations

Taking the impression
A study model impression should be taken using alginate (e.g. M+W Algicit) or silicone (e.g. M+W Sioplast A) before preparing for a crown or bridge or before a planned extraction. The interdental flash should be cut out to improve the stability of the temporary restoration to be fabricated at a later stage. In the case of alginate impressions, syringe the M+W Pontiform automix 2/ direct mix as soon as possible after taking the impression.

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Handling of double syringe
The double syringe automatically dispenses and mixes the material, which saves time when applying the luting material directly into the restoration. Remove the transport cap before using a new double syringe and then attach the mixing tip. Following application, the mixing tip acts as a seal and can be left on the double syringe until the next use. The mixing tip should only be replaced by a new tip when the double syringe is used again.

Cartridge handling
The material is dispensed and mixed automatically by extruding through a mixing tip. Remove the transport seal before using a new cartridge. Then attach the mixing tip and, if required, a conturation tip. Following use, the mixing tip should be used as a seal and should be left on the cartridge until the next application. The mixing tip should only be replaced with a new one when the cartridge is to be used again.

Recommended use and elastic phase
Dry the prepared teeth and coat the preparations, surrounding tissue and any existing acrylic with a separating agent Syringe M+W Pontiform automix 2/ direct mix into the deepest section of the impression and then up to the gingival areas. Always keep the mixing tip immersed in the material to avoid

bubbles. The M+W Pontiform automix 2/ direct mix has a firm, elastic consistency approximately 2 to 3 min after mixing and can be easily removed from the patient's mouth. The curing process must be monitored intraorally (e.g. with a probe), as the temporary restoration can only be easily removed during the elastic phase.

Final curing and preparation
M+W Pontiform automix 2/ direct mix cures fully in 3 min extraorally. The temporary restoration should be replaced in the impression during the final curing phase. Prepare and polish the temporary restoration after removing the oxygen inhibition layer with a solvent (e.g. alcohol). Commercially available temporary cements can be used for luting.

Repair of the temporary restoration
A feature of temporary restorations fabricated using M+W Pontiform automix 2/ direct mix is their high mechanical strength. If a temporary restoration should fracture however, we recommend the following procedure:

Fracture of the temporary restoration shortly after fabrication:
Bond the fracture sections with freshly mixed M+W Pontiform automix 2/ direct mix.

Fracture of an older or in situ temporary restoration:
Roughen the fractured sections and prepare undercuts. Bond the prepared fracture sections with freshly mixed M+W Pontiform automix 2/ direct mix. We recommend using a composite bond to improve the bond. Press the sections together for approx. 3 min.

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