ENGLISH

General Information

3M™ ESPE™ Filtek™ Bulk Fill Flowable Restorative, is a low viscously, visible-light activated, radiopaque, flowable. This low stress flowable material is semi-translucent enabling a 4 mm depth of cure. The restorative is packaged in capsules and syringes. The shades offered with Filtek Bulk Fill Flowable are U (Universal), A1, A2, and A3. Filtek Bulk Fill Flowable contains bisGMA, UDMA, bisEMA(6) and Photoclay resins. The fillers are a combination of ytterbium trifluoride filler with a combination of silica and zirconia with a particle size range of 0.01 to 3.5 µm. The inorganic filler loading is approximately 64.5% by weight (42.5% by volume). Filtek Bulk Fill Flowable is applied to the tooth following use of a compatible methacrylate-based dental adhesive, such as manufactured by 3M ESPE. Filtek Bulk Fill Flowable bonds the restoration to the tooth structure.

Indications

- Base under Class I and II direct restorations
- liner under direct restorative materials
- Pit and fissure sealant
- Restoration of minimally invasive cavity preparations (including small, non stress-bearing occlusal restorations)
- Class III and V restorations
- Undercut brookout
- Repair of small enamel defects
- Repair of small diastemas in esthetic indirect restorations
- Repair of resin and acrylic temporary materials
- As a core build-up where at least half the coronal tooth structure is remaining to provide structural support for the crown.

Precautionary Information for Patients
This product contains substances that may cause an allergic reaction by skin contact in certain individuals. Avoid use of this product in patients with known acrylate allergies. If prolonged contact with oral soft tissue occurs, flush with large amounts of water. If allergy reactions occur, discontinue use. If further treatment is needed, remove the product if necessary and discontinue future use of the product.

Precautionary Information for Dental Personnel
This product contains substances that may cause an allergic reaction by skin contact in certain individuals. To reduce the risk of allergic response, minimize exposure to these materials. In particular, avoid exposure to the product. If skin contact occurs, flush skin with soap and water. Use of protective gloves and a no-touch technique is recommended. Acrylates may penetrate commonly used gloves. If skin contact occurs, remove and discard glove, wash hands immediately with soap and water and then re-glove. If allergic reaction occurs, seek medical attention as needed.

3M ESPE MSDS information can be obtained from www.3MESP.com or contact your local sales representative.

Instructions for Use

Preparation

1. **Prophy:** Teeth should be cleaned with pumice and water to remove surface stains.
2. **Shade Selection:** Prior to isolation of tooth, select the appropriate shade(s) of Filtek Bulk Fill Flowable using a standard VITAPAN® classical shade guide. For sealants or crowns, a shade guide is not required.
Note: As Filtek Bulk Fill Flowable is a self-translucent, the location of the restoration, underlying tooth color or adjacent restorations may influence the final appearance of the liner/base.

3. **Isolation:** A rubber dam is the preferred method of isolation. Cotton rolls and an evacuator can also be used.

Directions

Band Preparation: Formation of tight proximal contacts is accomplished solely through careful matrix technique. Wedging is needed to provide a tight gingival seal and create a separation between teeth to compensate for thickness of the matrix band. Where possible, use of a precontoured sectional matrix band is preferred. However, if a standard band cannot be used, then apply the Filtek Bulk Fill Flowable. The band should be burnished to ensure proper proximal contour as Filtek Bulk Fill Flowable will not hold the band during placement.

2. **Pulp protection:** If a pulp exposure has occurred and the situation warrants a direct pulp capping procedure, use a minimum amount of calcium hydroxide on the exposure followed by application of 3M™ ESPE™ Vitrebond™ or Vitrebond Plus Light Cure Glass Ionomer. Vitrebond or Vitrebond Plus liners/bases may be used to line areas of deep cavity excavation.

3. **Adhesive System:** To bond Filtek Bulk Fill Flowable to tooth structure, use of a 3M ESPE dental adhesive system (for example 3M™ ESPE™ Scotchbond™ Universal) is recommended. Refer to adhesive system product instructions for full instructions and precautions for the products.

After curing the adhesive, continue to maintain isolation from blood, saliva and other fluids and proceed immediately to placement of Filtek Bulk Fill Flowable.
Note: Follow the adhesive system instructions for use for recommended silane treatment during repair of ceramic restorations followed by the adhesive application.

Delivery:

- Syringe delivery:** Filtek Bulk Fill flowable can be delivered directly from the dispensing tip.
 - Protective eyewear for patients and staff is recommended when using the dispensing tip.
 - Prepare the dispensing tip: Remove cap and save. Twist the prebent disposable dispensing tip assembly onto the syringe. Hold the tip to the tip from the patient and any dental staff, express a small amount of Filtek Bulk Fill flowable to assure that the delivery system is not plugged.
 - If plugged, remove the dispensing tip and express a small amount of Filtek Bulk Fill flowable restorative directly from the syringe. Remove any visible air bubbles by gently tapping the syringe against the dispensing tip and again express composite. Filtek Bulk Fill flowable restorative may be extruded onto a dispensing pad and applied with an appropriate instrument.
 - Place the functioning syringe with attached delivery tip into a suitably sized syringe cap and insert the cap into the syringe. Push the cap down the cannula for use. Using a barrier sheet facilitates cleaning and disinfection of the syringe between patients.

4. **Single-Dose Capsule:** Insert capsule into 3M™ ESPE™ Restorative Dispenser. Refer to separate restorative dispenser instructions for full instructions and precautions.

- Placement:**
 - Avoid intense light in the working field. Exposure of paste to intense light may cause premature polymerization.
 - Dispense Filtek Bulk Fill flowable: Start dispensing in the deepest portion of the preparation, holding the tip close to the preparation surface. As material is extruded, slowly raise the tip so it is slightly above the depth of the preparation. Filtek Bulk Fill flowable to assure that the delivery system is not plugged.
 - If plugged, remove the dispensing tip and express a small amount of Filtek Bulk Fill flowable restorative directly from the syringe. Remove any visible air bubbles by gently tapping the syringe against the dispensing tip and again express composite. Filtek Bulk Fill flowable restorative may be extruded onto a dispensing pad and applied with an appropriate instrument.
 - Place the functioning syringe with attached delivery tip into a suitably sized syringe cap and insert the cap into the syringe. Push the cap down the cannula for use. Using a barrier sheet facilitates cleaning and disinfection of the syringe between patients.
- Single-Dose Capsule:** Insert capsule into 3M™ ESPE™ Restorative Dispenser. Refer to separate restorative dispenser instructions for full instructions and precautions.

- Placement:**
 - Avoid intense light in the working field. Exposure of paste to intense light may cause premature polymerization.
 - Dispense Filtek Bulk Fill flowable: Start dispensing in the deepest portion of the preparation, holding the tip close to the preparation surface. As material is extruded, slowly raise the tip so it is slightly above the depth of the preparation. Filtek Bulk Fill flowable to assure that the delivery system is not plugged.
 - If plugged, remove the dispensing tip and express a small amount of Filtek Bulk Fill flowable restorative directly from the syringe. Remove any visible air bubbles by gently tapping the syringe against the dispensing tip and again express composite. Filtek Bulk Fill flowable restorative may be extruded onto a dispensing pad and applied with an appropriate instrument.
 - Place the functioning syringe with attached delivery tip into a suitably sized syringe cap and insert the cap into the syringe. Push the cap down the cannula for use. Using a barrier sheet facilitates cleaning and disinfection of the syringe between patients.
- Single-Dose Capsule:** Insert capsule into 3M™ ESPE™ Restorative Dispenser. Refer to separate restorative dispenser instructions for full instructions and precautions.

- Placement:**
 - Avoid intense light in the working field. Exposure of paste to intense light may cause premature polymerization.
 - Dispense Filtek Bulk Fill flowable: Start dispensing in the deepest portion of the preparation, holding the tip close to the preparation surface. As material is extruded, slowly raise the tip so it is slightly above the depth of the preparation. Filtek Bulk Fill flowable to assure that the delivery system is not plugged.
 - If plugged, remove the dispensing tip and express a small amount of Filtek Bulk Fill flowable restorative directly from the syringe. Remove any visible air bubbles by gently tapping the syringe against the dispensing tip and again express composite. Filtek Bulk Fill flowable restorative may be extruded onto a dispensing pad and applied with an appropriate instrument.
 - Place the functioning syringe with attached delivery tip into a suitably sized syringe cap and insert the cap into the syringe. Push the cap down the cannula for use. Using a barrier sheet facilitates cleaning and disinfection of the syringe between patients.
- Single-Dose Capsule:** Insert capsule into 3M™ ESPE™ Restorative Dispenser. Refer to separate restorative dispenser instructions for full instructions and precautions.

Shades	Increment depth	Cure Time	
		All halogen lights LED (with output 550-1000 mW/cm²)	Elipar™ S10 (3M™ ESPE™ LED with output 1000-2000 mW/cm²)
	U	4 mm	20 sec.
	A1, A2, A3	4 mm	40 sec.

- Complete the restoration:**
 - Base/Liner application: Place a composite restorative material, such as Filtek™ Supreme Ultra/Filtek™ Supreme XTE Ultra/Inform Restorative directly over the cured Filtek Bulk Fill flowable restorative. Follow the manufacturer’s instructions regarding curing, curing, finishing, occlusal adjustment, and polishing.
 - Pit and Fissure Sealant: Gently remove the inhibited layer remaining after light curing with syringe, push or polishing paste.
 - Core build-up:
 - The bulk fill restorative is compatible with commonly-used impressioning materials when surface inhibition layer is removed.
 - The bulk fill restorative should be kept wet with saliva or lubricated to prevent bonding to chemical-cure provisions.
 - Speedy temporary luting cements will not bond to the bulk fill restorative.
- Direct Restorative Application:**
 - Contour restoration surfaces with fine finishing diamonds, burs or stones.
 - Check occlusion with a thin articulating paper. Examine centric and lateral excusion contacts. Carefully adjust occlusion by removing material with a fine polishing diamond or stone.
- Polish with 3M™ ESPE™ Soft-Lex™ Finishing and Polishing System** or asseme. Ambient temperatures routinely higher than 27°C/80°F may reduce shelf life. See outer package for expiration date.

- Storage and Use**
 - This product is designed to be used at room temperature. If stored in cooler to protect to reach room temperature pur to use. Shelf life at room temperature is 36 months. Ambient temperatures routinely higher than 27°C/80°F may reduce shelf life. See outer package for expiration date.
 - Do not expose restorative materials to elevated temperatures, or to intense light.

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